KIT PRODUCT CATALOG

Osstem Implant 2016-17 Comprehensive Catalog

Overall Planning/Editing PR Department Design Team

Supervision Implant Lab, Marketing PM

Production/Distribution Marketing & Planning Team

Date of Publication 2016.Publisher Osstem Implant

8th FL, World Meridian II, 123, Gasan digital 2-ro,

Geumcheon-gu, Seoul, Korea

Phone +82.2.2016.7000

Fax +82.2.2016.7001

www.osstem.com

003 INTRODUCTION

O12 CONTENTSO20 OSSTEM KIT

118 REFERENCE



We are forever grateful to all the dentists who have given unwavering support to OSSTEM IMPLANT. We would like to take this opportunity to express our special thanks to all of our customers. Osstem has been continuously engaged in research and development, and over the past year our efforts have led to the launch of multiple new products.

We are excited to publish the 2016-17 Osstem Product Catalog, which showcases these new products and includes all the information you need to know about our fixtures, abutments, surgical tools and regenerative products. We spent countless hours editing, revising and rearranging this catalogue to make it the most comprehensive and user-friendly resource for Osstem products possible. Product codes, specifications and all the information you need about our products are at your fingertips.

A number of noteworthy additions have been introduced to the 2016 - 17 Osstem Product Catalog; each product is labeled with the newest products This is unclear. Also sounds impossible.

are at the highest resolution possible, and a comprehensive description is provided with each item to ensure customers order correctly. As well, the quality of the 2D and 3D graphical illustrations of our components has been significantly enhanced.

The color-coding system has been expanded from just fixtures to include components as well as tools. In addition, complimentary products and equipment are listed "complimentary" means "free of charge". "complementary" means "compatible". Unless you plan to give your products away you may want to change this.

Finally, the included QR codes link you to videos of the products being used in actual surgeries. It is our sincere hope that the 2016 - 17 Osstem Product Catalog will become an invaluable resource for you in your practice. Thank you!

How do you label a 'product with a product'?. All product images

CEO of OSSTEM IMPLANT
Choi Kyu-ok (DDS.Ph.D)

Olivilly of



1997

- **01** Established 'OSSTEM Co., Ltd.'
- 12 Released 'Doobunae' (health insurance claim application software program)

2001

- 01 Obtained CE-0434 certification
- 03 Established AIC Training Center
- **12** Established 12 overseas branches (first round)

certification (Russia)

company name to

04 Obtained GOST-R

Osstem Implant Co., Ltd

2008

- **01** Established Osstem Bone Science Research Center
- 12 Selected as a managing organization for the National Strategic Technology Development Project

2007

2006

01 Changed the

- **02** Listed on KOSDAQ and began trading publicly
- **06** Selected as No. 1 products for the next generation and obtained TGA certification (Australia)

2009

10 Obtained approval for medical device manufacturing and sale from the Ministry of Health, Labor and Welfare, Japan

2010

- 03 Launched TS III SA line 06 Launched TS III HA line
- **08** Selected as a business participating in the WPM Biomedical National Project
- 12 Users of Hanaro program exceeded 10,000

2011

- 10 Obtained 'Health Canada' certification
- 12 Launched 'K2 Unit Chair, which was selected as a 'World Class Product'

2013

- 01 Launched Osstem xenograft material 'A-Oss'
- 09 Launched 'K3 Unit Chair'
- 10 Selected as a 'Hidden Champion' company

2015

- 03 Established Osstem BioPharma Co., Ltd.
- 12 Awarded 'USD 50 Million Export Tower'

2000

- 06 Released 'Hanaro' (dentistry management software)
- 10 Acquired Sumin Comprehensive Dental Materials

2002

- **01** Established Osstem Implant R&D Center
- **08** Obtained FDA certification. launched USII line
- 10 Launched SSII line

2011

- **06** Osstem Implant R&D Center was selected as ATC (Advanced Technology Center) 07 Established Osstem
- 07 Selected as 'World Champ' business

2012


- **06** Launched TSIII CA line
 - Dental Equipment Research Institute

2014

- **05** Selected as World Class 300
- 05 Released 'HyFlex', an impression material
- 08 Released 'Beau TIS' whitening material

OSSTEM® Implant Design feature

OSSTEM IMPLANT has revolutionized implant dentistry in South Korea. With a focus on aggressive R&D, a commitment to education and a dedication to manufacturing the best products, Osstem Implant's ultimate goal is to become the global leader in implant dentistry.

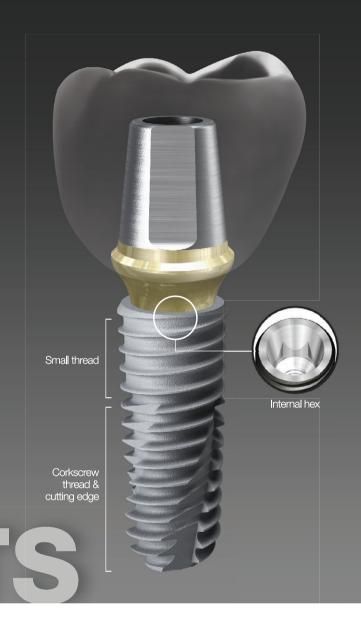









Each implant system has its own unique color code.

Submerged type implant with an internal hex and 11° tapered connection


- Internal connection type Mini / Regular
- Excellent initial stability in soft bone due to smaller threads in the upper section
- · Corkscrew thread with cutting edges
- Easy path adjustments via the implant's excellent self-threading effect
- Higher initial stability and consistent insertion torque, regardless of the drill's diameter
- Different body types to properly match the patient's bone quality and clinical condition
- TSII (straight body type): Easy to adjust the insertion depth
- TSIII (1.5° tapered body type): Excellent initial stability necessary for immediate loading, even in soft bone
- TSIV (6° tapered body type): Specifically designed for the maxillary sinus and soft bone, excellent initial stability
- Available surface types SA / CA / BA / HA

Non-submerged type implant with an internal octa and 8° tapered connection

- Internal connection type Regular / Wide
- Corkscrew thread with cutting edges
- Easy path adjustments via the implant's excellent self-threading effect
- Higher initial stability and consistent insertion torque, regardless of the drill's diameter
- Different body types to properly match the patient's bone quality and clinical condition
- SSII (straight body type): Easy to adjust the insertion depth
- SSIII (1.5° tapered body type): Excellent initial stability necessary for immediate loading, even in soft bone
- Available surface types SA / CA / HA

Submerged type implant with an external hex connection structure

- Internal connection type Mini / Regular / Wide / Wide PS
- Corkscrew thread with cutting edges
- Easy path adjustments via the implant's excellent self-threading effect
- Higher initial stability and consistent insertion torque, regardless of the drill's diameter
- Different body types to properly match the patient's bone quality and clinical condition
- USII (straight body): Easy to adjust the insertion depth
- USIII (1.5° tapered body): Excellent initial stability necessary for immediate loading, even in soft bone
- USIV (6° tapered body): Specifically designed for the maxillary sinus and soft bone, excellent initial stability
- Available surface types SA / CA

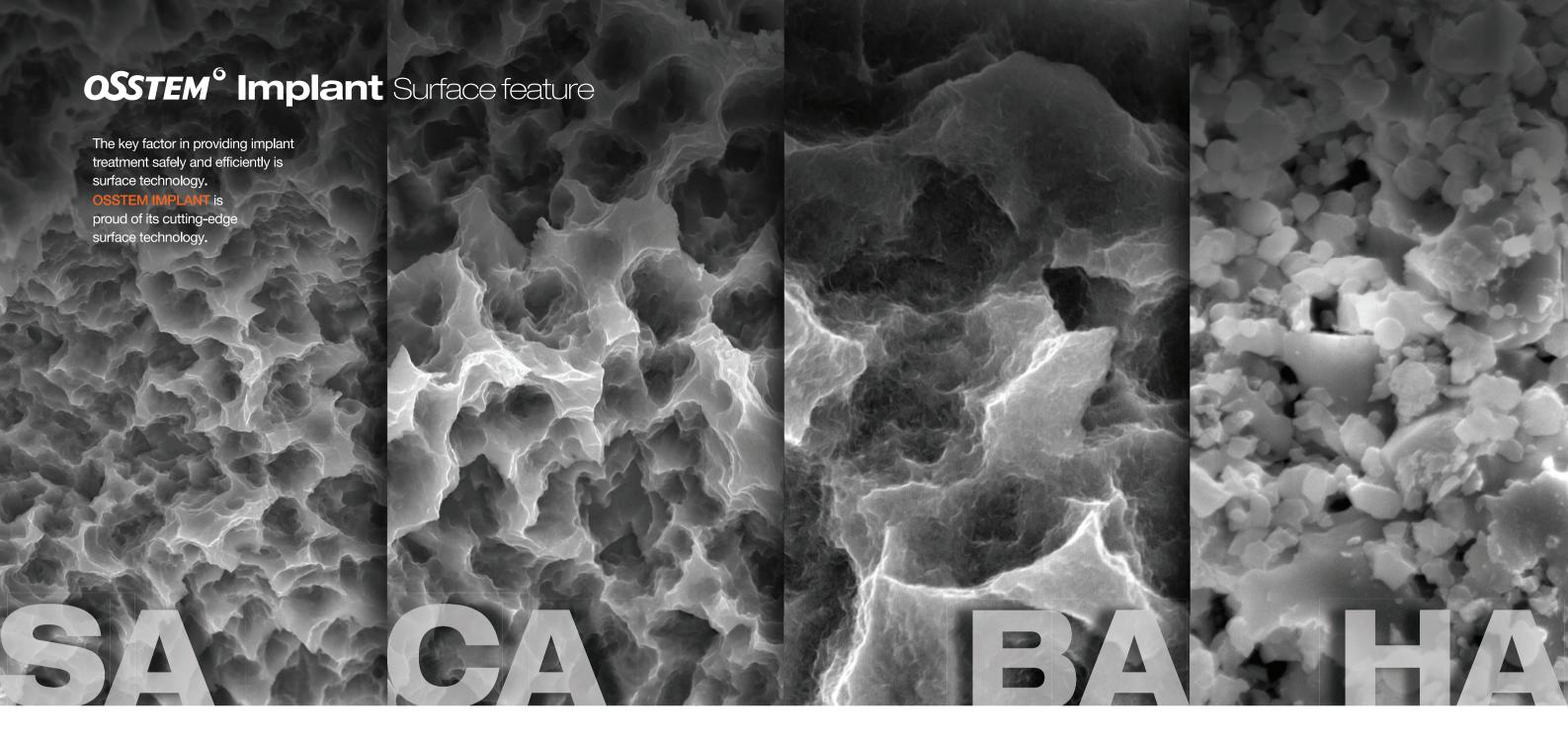

Acid Treated Optimized Surface

- Surface roughness value of about Ra 2.5 to $3.0 \mu m$ Note: The upper 0.5mm section of the implant has a surface roughness of about Ra 0.5 to $0.6 \mu m$ Consistent surface micro pits between 1 to $3 \mu m$
- Surface area is increased by 46 percent compared to RBM treated implants

In Bone Response

- · Increased osteoblastic differentiation and ossification by 20 percent compared to RBM surface
- Initial bone response studied in animal models (mini pigs)
 Initial stability increased initial stability by
- 48 percent compared to RBM surface (RT at 4 weeks)
- Ossification rate increased ossification rate by 20 percent compared to RBM surface (BIC at 4 weeks)

Super-hydrophilic SA surface suspended in a calcium solution

- · Same SA surface morphology
- Optimizing surface reaction by suspension in a calcium (CaCl2) solution
- Increased new bone formation area due to the excellent blood wettability
- Bone response improved in early osseointegration stage compared to standard SA surface

In Bone Response

- Protein and cellular adhesion tripled compared to
 SA surfaces.
- Initial cellular differentiation by 19 percent compared to SA surfaces (7 days)
- Initial stability increased by 34 percent compared to SA surfaces (RT at 4 weeks)
- Ossification rate Increased by 26 percent compared to SA surfaces (BIC at 4 weeks)

Premium low crystalline nano-HA coated SA surface

- \cdot SA surface (Ra 2.5 to 3.0 μ m) coated with HA
- · 10nm ultra-thin HA coating
- · Dual function between titanium and HA
- HA is naturally resorbed during ossification

Premium high-crystalline HA-coated surface

- · 30 to 60 µm thick high-crystalline HA coating
- · HA coated onto a RBM surface (Ra 3.0 to 3.5μm)
- · High HA crystalline over 98 percent
- · Solved the problem with low-crystalline HA resorption

In Bone Response

- · Advantages of both SA and HA surfaces
- SA's ability to maintain an optimal surface morphology
- HA's ability to form high quality initial bone, even in a poor bone quality
- Ossification rate increased by 40 percent compared to SA surfaces (BIC)
- · Unlike conventional HA surfaces, it is applicable to all types of bone quality

In-vitro & In-vivo Bone Response

- Excellent biocompatibility because HA is very similar to actual human bone
- · Initial ossification by osteoblasts doubled compared to SA surfaces (5 days)
- Initial stability in animal models increased by
 40 percent compared to SA surfaces (RT, 4 weeks)
- Suitable for poor bone quality, tooth extraction sites or immediate implant insertion

KIT Contents 1/3



KIT Contents 2/3



KIT Contents 3/3










O21 Taper KIT
O22 Taper Ultra KIT
O23 123 KIT
O24 123 Full KIT
O25 123 KIT - IV Type
O26 Ultra KIT
O28 New Hanaro KIT
O50 Prosthetic KIT
O58 CAS KIT
O61 CAS KIT Plus
O62 LAS KIT Plus
O65 LAS KIT Plus
O66 ESSET KIT
O70 MS KIT

020 122 Taper KIT


O71 Ortho KIT
O72 Bone Screw KIT
O73 Custom KIT
O74 Osteo KIT
O75 Osteotome KIT
O76 Sinus KIT
O77 Bone Spreader KIT
O78 Ridge Split KIT - Straight
O79 Ridge Split KIT - Offset
O80 ESR KIT
O84 EFR KIT
O87 SlowJec
O88 Parallel Guide KIT
O92 SmartGuide KIT

122 Taper KIT (0122TPK)

For TSIII SSIII USIII Implant Systems

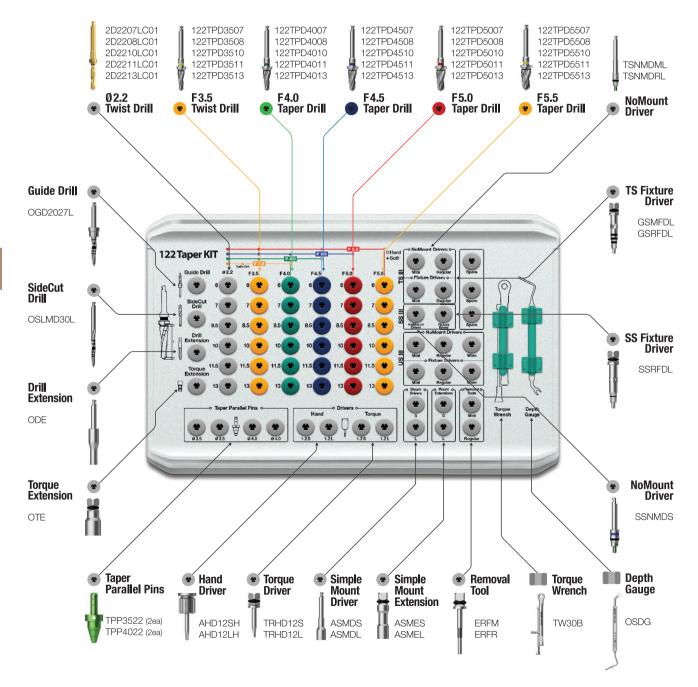



Taper KIT (OTSK)

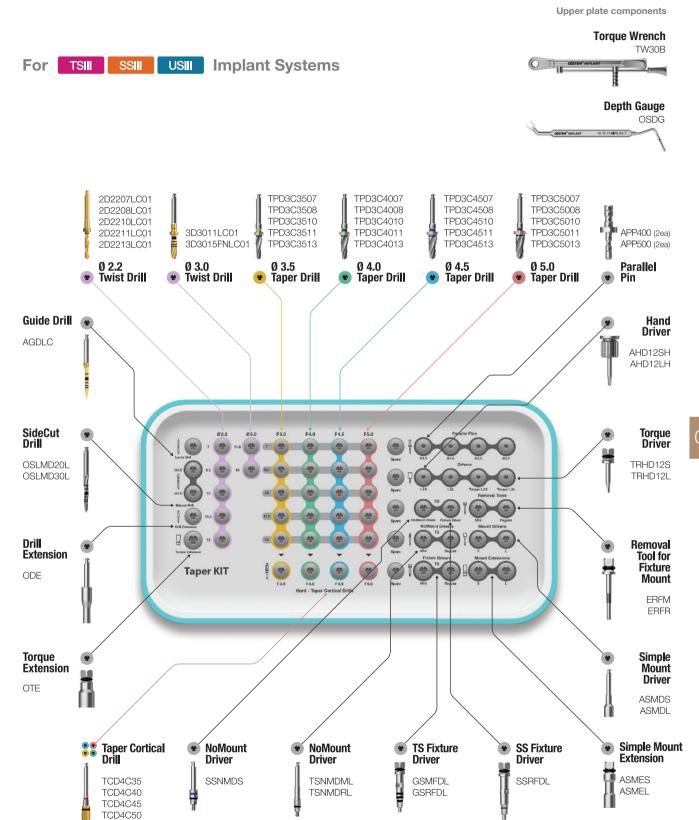

Taper Ultra KIT (HULTPK)


For **III Ultra-wide** Implant Systems

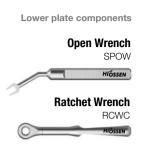

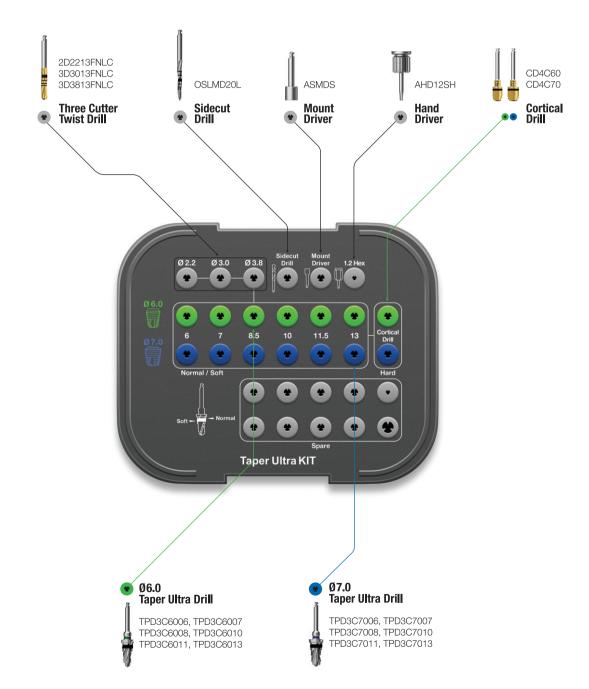


123 KIT (0123K)


For TSII/III SSII/III USII/III Implant Systems

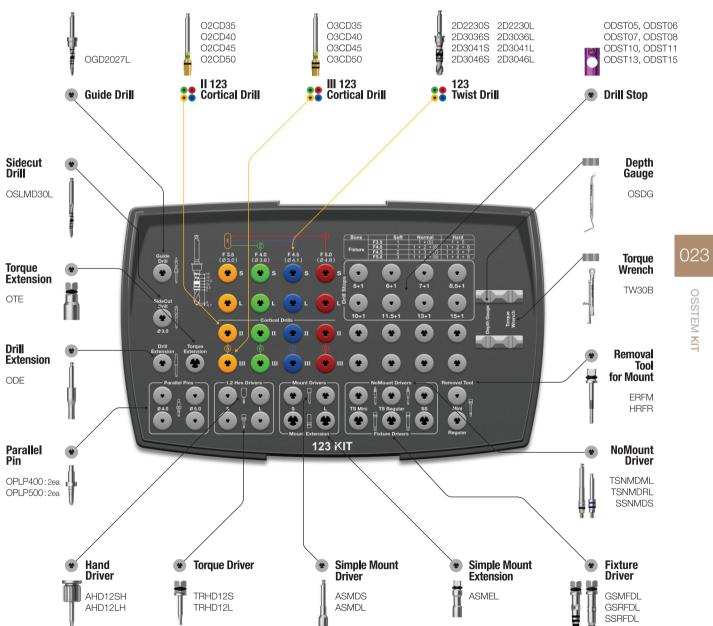

TSII/III SSII/III USII/III Implant Systems

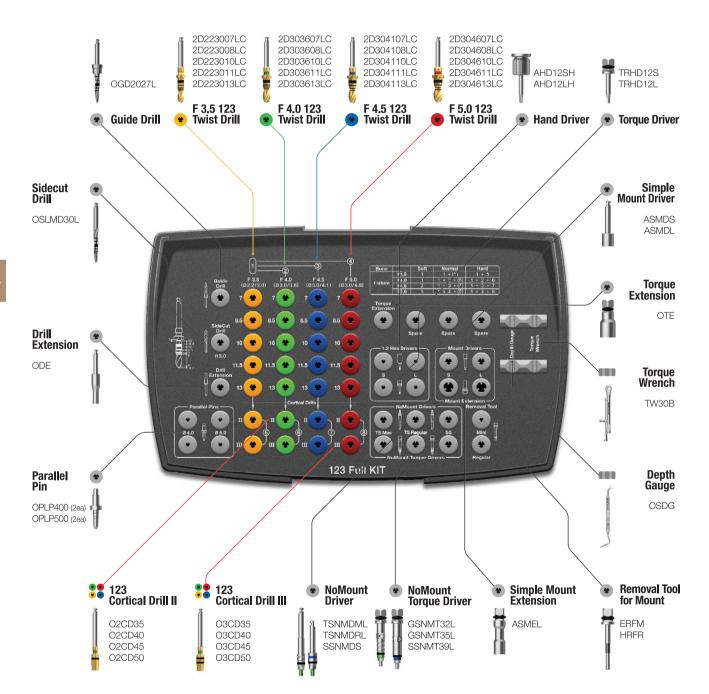


For TSIV USIV Implant Systems

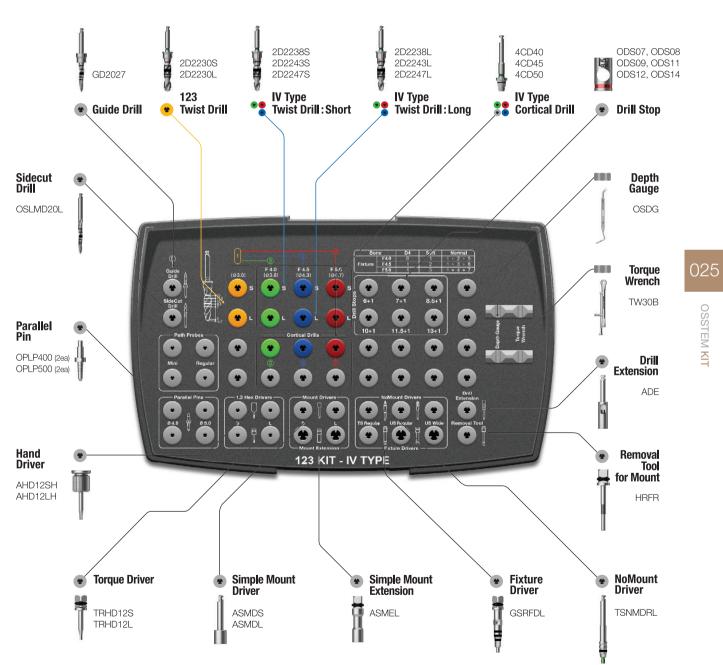


Lower plate components

New Hanaro KIT (HKA2)

For TSII/III SSII/III USII/III Implant Systems

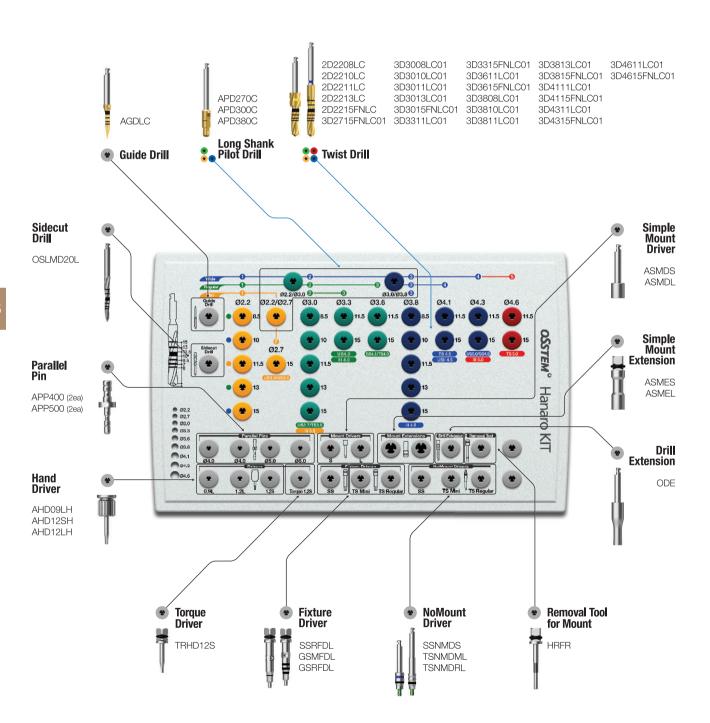


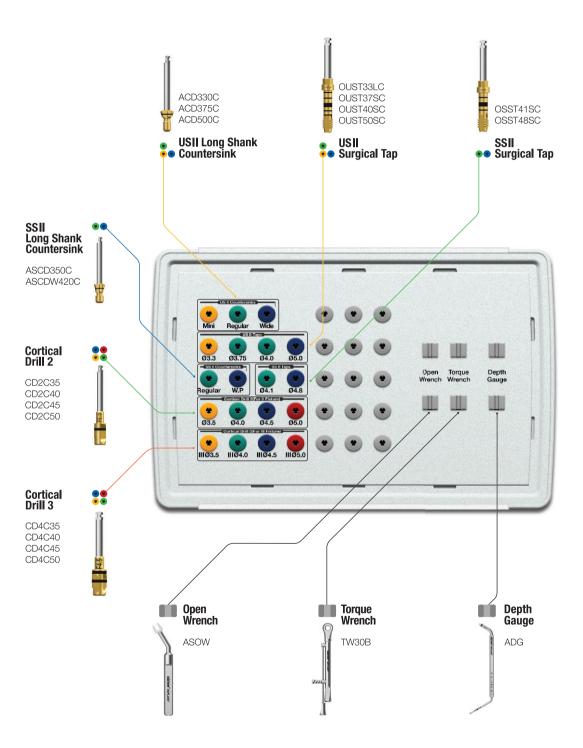

Lance Drill - Guide Drill

- Used to create an initial osteotomy in the bone to facilitate initial drilling
- · Bone density can be determined by drilling

| \ L | Short | Long |
|-----|-------|-------|
| | AGDSC | AGDLC |

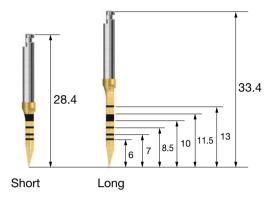

123 Guide Drill

- Used to create an initial osteotomy in the bone to facilitate initial drilling
- Easy drill depth control by selecting the appropriate drill stopper






Sidecut Drill

- Capable of side cutting an osteotomy using the drill body's cutter blades
- For trimming the ridge of an extraction socket
- Facilitating site preparation of an extraction socket

| L D1/D2 | Ø1.5/2.0 | Ø2.0/2.5 | Ø2.5/3.0 | Ø3.0/3.5 |
|---------|----------------|------------------|------------------|------------------|
| 13 | OSLM DS | OSLMD 20S | OSLMD 25S | OSLMD 30S |
| 16.5 | - | - | OSLMD 25L | OSLMD 30L |
| 20 | OSLM DI | OSLMD 20I | _ | _ |

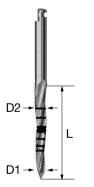

Drill Extension


- Extends the length of drills and other hand-piece tools
- Make sure that it is securely fastened to prevent bending or breaking due to excessive force.
- Extends the length of a drill by 16.9mm





Twist Drill - Stopper Drill


- Long stopper (6mm)
- Posterior drilling possible without an extension
- The color-coded by length

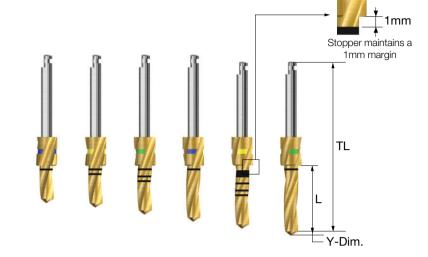


| L _ | TL D | Ø2.2 | Ø3.0 | Ø3.3 | Ø3.6 | Ø3.8 | Ø4.1 | Ø4.3 | Ø4.6 |
|------|--------|--------------------|--------------------|--------------------|------------|--------------------|--------------------|--------------------|--------------------|
| | Y-Dim. | 0.6 | 0.9 | 1.0 | 1.0 | 1.0 | 1.0 | 1.0 | 1.0 |
| 6.0 | 30.5 | 2D22 06LC | 3D30 06LC | - | - | 3D38 06LC | = | = | - |
| 7.0 | 31.5 | 2D22 07LC01 | 3D30 07LC01 | - | - | 3D38 07LC01 | - | - | - |
| 8.5 | 33 | 2D22 08LC01 | 3D30 08LC01 | - | = | 3D38 08LC01 | - | = | - |
| 10.0 | 34.5 | 2D22 10LC01 | 3D30 10LC01 | - | - | 3D38 10LC01 | - | - | - |
| 11.5 | 34.5 | 2D22 11LC01 | 3D30 11LC01 | 3D33 11LC01 | 3D3611LC01 | 3D38 11LC01 | 3D41 11LC01 | 3D43 11LC01 | 3D46 11LC01 |
| 13.0 | 36 | 2D22 13LC01 | 3D30 13LC01 | - | - | 3D38 13LC01 | - | - | - |

Twist Drill - Non Stopper Drill

- Used when adjacent teeth prevent the use of the Stopper Drill (tight spaces).
- Short and long laser marked drills are available
- Please see the Non Stopper Drill images for marking measurements

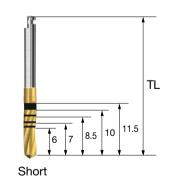


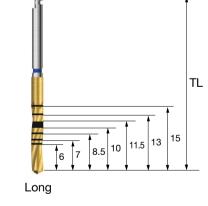

| TL\ <u>D</u> | Ø1.5 | Ø2.0 | Ø2.2 | Ø2.7 | Ø3.0 | Ø3.3 |
|--------------|--------------------|--------------------|--------------------------------|--------------------------------|--------------------------------|----------------------|
| 33 | 2D15 18FNLC | 2D20 18FNLC | 2D22 18FNLC | 3D27 18FNLC | 3D30 18FNLC | 3D33 18FNLC |
| 41 | - | - | 2D22 15FNLC01 | 3D27 15FNLC01 | 3D30 15FNLC01 | 3D33 15FNLC01 |
| | | | | | | |
| TL\D | Ø3.6 | ø3.8 | Ø4.1 | Ø4.3 | Ø4.6 | |
| TL <u>D</u> | Ø3.6 3D3618FNLC | Ø3.8 3D3818FNLC | Ø4.1 3D41 18FNLC | Ø4.3 3D43 18FNLC | Ø4.6 3D46 18FNLC | |

123 Twist Drill

- Unique straight drill that reduces the drilling sequence
- Color codes indicate the drill's diameter and the matching fixture to be placed
- Controll drilling depth by selecting the appropriate drill stopper
- CAUTION: The drill stopper must be used, it can be difficult to control the drilling depth
- F = Fixture

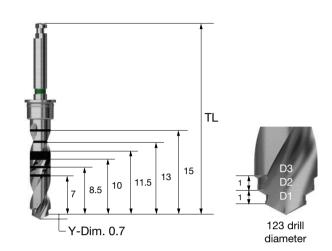

| | D1/D2/D3 | | | | | |
|-------|----------------|----------------|--------------------|--------------------|--|--|
| TL\ | F3.5(Ø2.2/3.0) | F4.0(Ø3.0/3.6) | F4.5(Ø3.0/3.6/4.1) | F5.0(Ø3.0/4.1/4.6) | | |
| 34 | 2D2230S | 2D3036S | 2D3041S | 2D3046S | | |
| 40.4 | 2D2230L | 2D3036L | 2D3041L | 2D3046L | | |
| Color | Yellow | Green | Blue | Red | | |


123 Drill Stopper

- The indicated length of the drill stopper is the actual drillable length before reaching the stopper
- Color-coded stopper identify the length and place in the kit

| L | 6.2 | 7 | 8 | 9.5 | 11 | 12.5 | 14 | 16 |
|---|--------|--------|--------|--------|--------|--------|--------|--------|
| | [5] | [©] | | 8.5 | 10 | 41. | | 15 |
| | ODST05 | ODST06 | ODST07 | ODST08 | ODST10 | ODST11 | ODST13 | ODST15 |

123 Twist Drill - Stopper Drill


- Straight drill that reduces the drilling sequence (stopper included)
- Color codes indicate the drill's diameter and the matching fixture to be placed
- F = Fixture

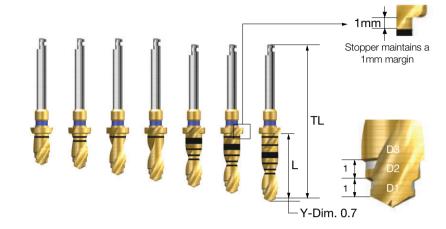

D1/D2/D3 F3.5(Ø2.2/3.0) F4.0(Ø3.0/3.6) F4.5(Ø3.0/3.6/4.1) F5.0(Ø3.0/4.1/4.6) L TL 6.0 30.5 2D2230**06LC** 2D3036**06LC** 2D3041**06LC** 2D3046**06LC** 7.0 31.5 2D2230**07LC** 2D3036**07LC** 2D3041**07LC** 2D3046**07LC** 8.5 33 2D2230**08LC** 2D3036**08LC** 2D3041**08LC** 2D3046**08LC** 10.0 34.5 2D2230**10LC** 2D3036**10LC** 2D3041**10LC** 2D3046**10LC** 11.5 34.5 2D2230**11LC** 2D3036**11LC** 2D3041**11LC** 2D3046**11LC** 13.0 36 2D2230**13LC** 2D3036**13LC** 2D3041**13LC** 2D3046**13LC** 15.0 38 2D3036**15LC** 2D3041**15LC** 2D3046**15LC** 2D2230**15LC** Color Yellow Green Blue Red

IV Type Twist Drill

- Drill specifically for placing type IV fixtures
- Color codes indicate the drill's diameter and the matching fixture to be placed
- The drill stopper must be used, it can be difficult to control the drilling depth
- F = Fixture

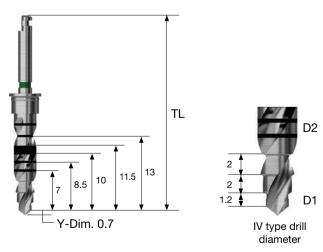


| | | D1/D2 | | |
|-------|-----------------|-----------------|-----------------|--|
| TL\ | F4.0 (Ø2.2/3.8) | F4.5 (Ø2.2/4.3) | F5.0 (Ø2.2/4.7) | |
| 34 | 2D2238S | 2D2243S | 2D2247S | |
| 40.4 | 2D2238L | 2D2243L | 2D2247L | |
| Color | Green | Blue | Red | |

OSSTEM

Surgical Instruments

Direct Drill


- Direct drill: Two-step drill that functions like a pilot and twist drill
- Final drilling is possible without using pilot drilling
- Increases initial stability in an extraction socket due to the reduced dead space at the apex

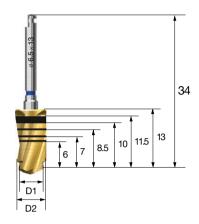

| D1/D2 | Ø4.6/5.2 | Ø4.6/5.5 | Ø5.5/6.2 | Ø5.5/6.5 |
|-------|------------|------------|------------|------------|
| | 3D5213FNLC | 3D5513FNLC | 3D6213FNLC | 3D6513FNLC |

122 Taper Drill


- Drill specifically for tapered (Type III) fixtures
- Range of diameters and lengths are available
- Color codes indicate the drill's diameter and the matching fixture to be placed
- For hard bone cases, use a drill one diameter bigger
- F = Fixture

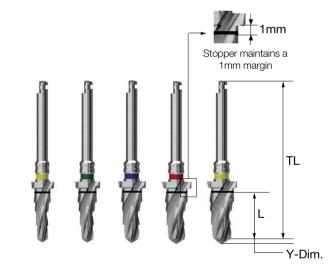

| L | TL | F3.5 | F4.0 | F4.5 | F5.0 | F5.5 |
|-------|--------|--------------------|--------------------|--------------------|--------------------|--------------------|
| | Y-Dim. | 0.7 | 0.9 | 1.0 | 1.0 | 1.0 |
| 4.0 | 29.5 | 122TPD 3504 | 122TPD 4004 | 122TPD 4504 | 122TPD 5004 | 122TPD 5504 |
| 5.0 | 29.5 | 122TPD 3505 | 122TPD 4005 | 122TPD 4505 | 122TPD 5005 | 122TPD 5505 |
| 6.0 | 30.5 | 122TPD 3506 | 122TPD 4006 | 122TPD 4506 | 122TPD 5006 | 122TPD 5505 |
| 7.0 | 31.5 | 122TPD 3507 | 122TPD 4007 | 122TPD 4507 | 122TPD 5007 | 122TPD 5507 |
| 8.5 | 33 | 122TPD 3508 | 122TPD 4008 | 122TPD 4508 | 122TPD 5008 | 122TPD 5508 |
| 10.0 | 34.5 | 122TPD 3510 | 122TPD 4010 | 122TPD 4510 | 122TPD 5010 | 122TPD 5510 |
| 11.5 | 34.5 | 122TPD 3511 | 122TPD 4011 | 122TPD 4511 | 122TPD 5011 | 122TPD 5511 |
| 13.0 | 36 | 122TPD 3513 | 122TPD 4013 | 122TPD 4513 | 122TPD 5013 | 122TPD 5513 |
| 15.0 | 38 | 122TPD 3515 | 122TPD 4015 | 122TPD 4515 | 122TPD 5015 | 122TPD 5515 |
| Color | | Yellow | Green | Blue | Red | Yellow |

Taper Drill

- Drill specifically for tapered (Type III) fixtures, comes in different diameters and lengths
- Stopper drill maintains a 1mm margin
- Color codes indicate the drill's diameter and the matching fixture to be placed
- F3.5: yellow, F4.0: green, F4.5: blue, F5.0: red, F5.5: yellow

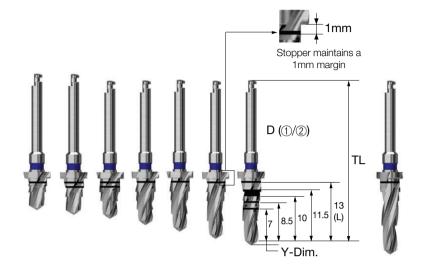


| L \ | TL | F3.5 | F4.0 | F4.5 | F5.0 | F5.5 |
|-------|--------|-------------------|-------------------|-------------------|-------------------|-------------------|
| | Y-Dim. | 0.8 | 0.9 | 1.0 | 1.0 | 1.0 |
| 5.0 | 29.5 | TPD3C 3505 | TPD3C 4005 | TPD3C 4505 | TPD3C 5005 | = |
| 6.0 | 30.5 | TPD3C 3506 | TPD3C 4006 | TPD3C 4506 | TPD3C 5006 | TPD3C 5506 |
| 7.0 | 31.5 | TPD3C 3507 | TPD3C 4007 | TPD3C 4507 | TPD3C 5007 | TPD3C 5507 |
| 8.5 | 33 | TPD3C 3508 | TPD3C 4008 | TPD3C 4508 | TPD3C 5008 | TPD3C 5508 |
| 10.0 | 34.5 | TPD3C 3510 | TPD3C 4010 | TPD3C 4510 | TPD3C 5010 | TPD3C 5510 |
| 11.5 | 34.5 | TPD3C 3511 | TPD3C 4011 | TPD3C 4511 | TPD3C 5011 | TPD3C 5511 |
| 13.0 | 36 | TPD3C 3513 | TPD3C 4013 | TPD3C 4513 | TPD3C 5013 | TPD3C 5513 |
| 15.0 | 38 | TPD3C 3515 | TPD3C 4015 | TPD3C 4515 | TPD3C 5015 | TPD3C 5515 |
| Color | | Yellow | Green | Blue | Red | Yellow |

Taper Ultra Drill

- \bullet Drill specifically for tapered ultra-wide fixtures, comes in different diameters and lengths
- Stopper drill maintains a 1mm margin
- Color codes indicate the drill's diameter and the matching fixture to be placed
- F = Fixture

| L | F6.0 | F7.0 |
|-------|-------------------|-------------------|
| 6 | TPD3C 6006 | TPD3C 7006 |
| 7 | TPD3C 6007 | TPD3C 7007 |
| 8.5 | TPD3C 6008 | TPD3C 7008 |
| 10 | TPD3C 6010 | TPD3C 7010 |
| 11.5 | TPD3C 6011 | TPD3C 7011 |
| 13 | TPD3C 6013 | TPD3C 7013 |
| Color | Green | Blue |

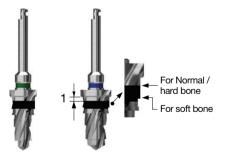

Long Shank Pilot Drill

- Corrects the drilling path
- Maintains the path of the previous drilling sequence

D1/D2 Ø2.0/2.7 Ø2.0/3.0 Ø3.0/3.8 Ø3.0/4.1


APD380C



123 Cortical Drill

- Trims cortical bone in hard bone cases
- · Recommend drilling until reaching the bottom of the marker
- Type II cortical drill marker is for hard bone
- Type III cortical drill bottom marker is for normal bone, the upper for hard bone
- Type IV cortical drill marker is for normal bone
- Color codes indicate the drill's diameter and the matching fixture to be placed
- F = Fixture

| Гуре | F3.5 | F4.0 | F4.5 | F5.0 |
|-------|----------------|----------------|----------------|----------------|
| I | 02CD 35 | O2CD 40 | O2CD 45 | O2CD 50 |
| II | 03CD 35 | 03CD 40 | O3CD 45 | 03CD 50 |
| V | - | 4CD 40 | 4CD 45 | 4CD 50 |
| Color | Yellow | Green | Blue | Red |

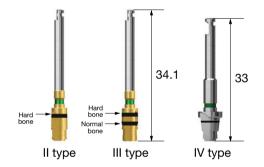

Cortical Drill 2 for TSII, SSII SA

- Trims cortical bone in hard bone cases (for type II)
- Drill specifically for Type II fixture's unique diameter
- Recommend drilling until reaching the bottom of the marker
- F = Fixture

| F3.5 | F4.0 | F4.5 | F5.0 |
|--------|--------|--------|--------|
| CD2C35 | CD2C40 | CD2C45 | CD2C50 |



Cortical Drill 3 for Taper Fixture (TSIII, SSIII, USIII)

- Use after straight drill to expand cortical bone
- In normal to hard bone, used as the final drill
- Drill specifically for Type III fixture's unique diameter
- The lower marker is for normal bone, the upper is for hard bone
- · Recommend drilling until reaching the bottom of the marker

F3.0 F3.5 F4.0 F4.5 F5.0 F5.5

CD4C30 CD4C35 CD4C40 CD4C45 CD4C50 CD4C55

Taper Cortical Drill for Taper Fixture (TSIII, SSIII, USIII)

- Trims cortical bone in hard bone cases (immediately after the taper drill)
- Drill specifically for Type III fixture's unique diameter
- F3.5~5.0 drill's lower marker is for fixture lengths ≤8.5mm and the upper is for lengths ≥10mm
- F5.5 drill's lower marker is for fixture lengths ≤6mm, the middle is for \leq 7mm, and the upper is for \geq 8.5mm
- Recommend drilling until reaching the bottom of the marker
- F = Fixture


F3.5 F4.0 F4.5 F5.0 F5.5

TCD4C35 TCD4C40 TCD4C45 TCD4C50 TCD4C55

Cortical Drill for Ultra-Wide

- Trims cortical bone in hard bone cases (for ultra-wide type fixtures)
- Drill specifically for Ultra-wide Type fixture's unique diameter
- Recommend drilling until reaching the bottom of the marker
- F = Fixture

| F6.0 | F7.0 | |
|--------|--------|--|
| CD4C60 | CD4C70 | |

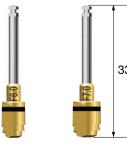




Countersink for USIII, USII SA, USIII SA (Wide PS, Wide)

- Drill specifically for USIII, USII SA, and USIII SA Wide PS and Wide type fixtures
- Recommended drilling speed: 300rpm



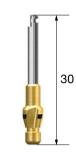

Straight Fixture Tap for TSII, USII, SSII SA

- Tap for straight body fixtures (type II)
- For hard bone, taps osteotomy creating fixture thread shape
- Recommended speed: 25rpm or hand torque
- Recommended tapping until reaching the bottom of the marker
- F = Fixture


| F3.5 | F4.0 | F4.5 | F5.0 | |
|---------|---------|---------|---------|--|
| O2FTS35 | O2FTS40 | O2FTS45 | O2FTS50 | |



Tapered Fixture Tap for TSIII, USIII, SSIII SA

- Tap for tapered body fixture (type III)
- \bullet For hard bone, taps osteotomy creating fixture thread shape
- Recommended speed: 25rpm or hand torque
- Recommended tapping until reaching the bottom of the marker (F5.0, the lower marker is for fixture lengths \leq 7.0mm and the upper is for \geq 8.5mm)
- F = Fixture


| F3.5 | F4.0 | F4.5 | F5.0 |
|--------|--------|--------|--------|
| OFTS35 | OFTS40 | OFTS45 | OFTS50 |

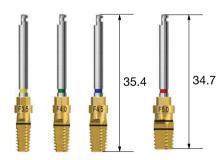

Parallel Pin

· Identifies the direction and location of the osteotomy


| D | Ø4.0 | Ø5.0 | Ø6.0 | Full Set |
|---|--------|--------|--------|----------|
| | APP400 | APP500 | APP600 | APPS |

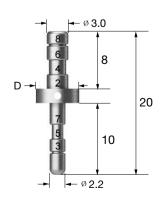

Parallel Pin for 123 Drill

- Parallel pin for the 123 twist drill
- · Identifies the direction and location of the osteotomy
- The lower half is use after initial drilling, upper half is for after F3.5 (Ø 2.2/3.0) drilling

| D | Ø4.0 | Ø5.0 | |
|---|---------|---------|--|
| | OPLP400 | OPLP500 | |

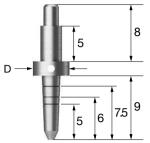


Parallel Pin for Taper Drill

- Parallel pin for the taper drill
- Identifies the direction and location of the osteotomy
- The lower half check after standard drilling, upper half is for after initial drilling
- Color-codes indicate fixture diameter

(F3.5 : yellow, F4.0 : green, F4.5 : blue, F5.0 : silver)

| F3.5 | F4.0 | F4.5 | F5.0 |
|---------|---------|---------|---------|
| TPP3522 | TPP4022 | TPP4522 | TPP5022 |

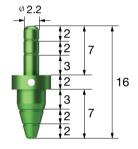

Depth Gauge

- A: Measures drilling depth (7~15mm)
- \bullet B: Measures the height of the gingiva after placing the fixture





Depth Gauge

• Measures drilling depth (7 to 15mm) and works as an open wrench







Trial Pin for Ultra-wide

- Measures the width and depth of a failed implant site
- Measure the drilling depth after using the Direct Drill as the final drill
- · Also serves as a parallel pin

D Ø5.2 Ø5.5 Ø6.2 Ø6.5

UWFTP52 UWFTP55 UWFTP62 UWFTP65

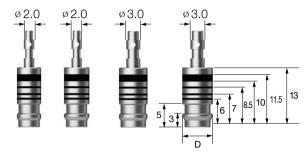

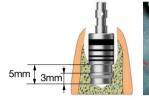


Minimum fastening height necessary for insertion right after extraction

Positioning Guide

- Sets the drilling interval for fixture insertion
- Insert after initial drilling
- Packing unit: The components and packages



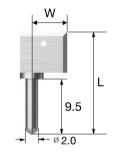

Tissue Height Gauge for TS

• Connects to the TS fixture to measure the height of the gingiva in relation to the fixture





Ratchet Wrench

- Anti-backlash mechanism prevents wrench from backdriving
- Note: Excessive torquing may cause damage to the bone or the inside of a fixture




CITQW-1185A

L-Wrench

- 1.2 hex driver for hard to reach areas like narrow intermaxillary areas
- Torque indication: When the wrench starts to bend (around 10°), apply 5 to 8Ncm of torque



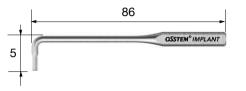

Torque Wrench - Spring Type

- Applies a precise amount of torque (10/20/30Ncm) to the screw and abutment
- The neck of the torque wrench will bend when the exact amount of torque has been delivered.
- Do not continue to torque after the neck has bent. Excessive force may cause damage to the implant, screw, abutment and hard tissue.





Torque Wrench - Bar Type

- · Adjusts the implant depth, and tightens abutments, screws, etc.
- Pull the bar back until reaching the desired torque value





Torque Wrench Set

- Bi-directional torque wrench (a torque connector is included)
- Without separating the connector, rotate the handle to apply torque, either in a forward or a backward direction
- Compatible with Osstem's machine driver connector
- Pull the bar back until reaching the desired torque value
- Packing unit : changeable torque wrench + torque connector





Anterior Hand Driver for Implant

- Manually torque implants in the anterior area
- Connect to a NoMount torque driver or a fixture driver
- Excessive torquing may cause damage to the bone or the inside of a fixture

AHDI



Torque Handle

- Connect with a contra-angle hand piece (set hand piece gear ratio to 1:1)
- Connects healing abutments, cover screws, abutment screws, orthodontic screws, etc.

(Note: After connecting the part, make sure that it is tightened with a torque wrench)

• Excessive torquing may cause damage to the screw fracture or hand piece


TQHD



NoMount Driver for TS

- Engine driver connects directly with the fixture (without a mount) for placement.
- C = Connection


| L\C | Mini | Regular |
|---------|---------|---------|
| Short | TSNMDMS | TSNMDRS |
| Long | TSNMDML | TSNMDRL |
| Ex.Long | TSNMDME | TSNMDRE |

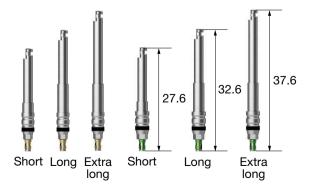

NoMount Driver for SS

- Engine driver connects directly with the fixture (without a mount) for placement.
- C = Connection


| L\C | Regular/Wide |
|---------|--------------|
| Short | SSNMDS |
| Long | SSNMDL |
| Ex.Long | SSNMDE |

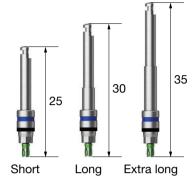

NoMount Driver for US

- Engine driver connects directly with the fixture (without a mount) for placement.
- C = Connection


| L\C | Mini | Regular | Wide |
|-------|-----------|-----------|-----------|
| Short | USNMD35MS | USNMD41RS | USNMD51WS |
| Long | USNMD35ML | USNMD41RL | USNMD51WL |

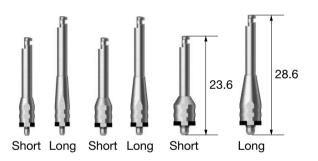

NoMount Torque Driver for TS

- Torque wrench driver connects directly with the fixture (without a mount) for placement.
- Make sure fixture and diver is securly connected; loose connection may cause fixture fracture.
- \cdot C = Connection

| L C | Mini | Regular |
|---------|----------|----------|
| Short | GSNMT32S | GSNMT35S |
| Long | GSNMT32L | GSNMT35L |
| Ex.Long | GSNMT32E | GSNMT35E |

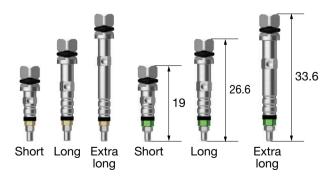

NoMount Torque Driver for SS

- Torque wrench driver connects directly with the fixture (without a mount) for placement.
- Make sure the fixture and the driver are securely connected; loose connection may cause fixture fracture.
- C = Connection


| L \ C | Regular/Wide | |
|-------|--------------|--|
| Short | SSNMT39S | |
| Long | SSNMT39L | |

Short Long

Fixture Driver for TS

- Connects directly to the fixture for final adjustments to the implant's depth. Also removes the implant.
- C = Connection


| L\C | Mini | Regular |
|---------|--------|---------|
| Short | GSMFDS | GSRFDS |
| Long | GSMFDL | GSRFDL |
| Ex.Long | GSMFDE | GSRFDE |

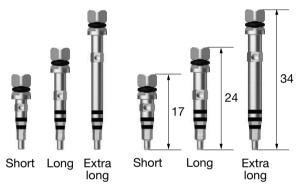

Fixture Driver for SS

- Connects directly to the fixture for final adjustments to the implant's depth. Also removes the implant.
- C = Connection

| L \ C | Regular/Wide |
|---------|--------------|
| Short | SSRFDS |
| Long | SSRFDL |
| Ex.Long | SSRFDE |

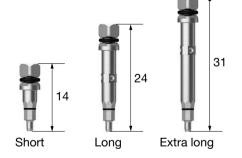

Fixture Driver for US

- Connects directly to the fixture for final adjustments to the implant's depth. Also removes the implant.
- C = Connection

| / C | Mini | Regular | Wide |
|-----|--------|---------|--------|
| | USMFDL | USRFDL | USWFDL |







Torque Extension

• Extends the length of an instrument by 10mm


OTE



Simple Mount Driver

• Connects to mounted fixtures for placement


Short ASMDS Long ASMDL Ex.Long **ASMDE**

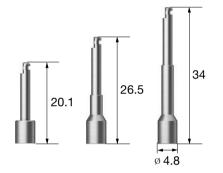

Simple Mount Extension

• Extends the length the Simple Mount Driver

Short ASMES Long **ASMEL**

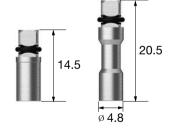

Simple Open Wrench

- Disengages the simple mount when bone quality is poor
- The neck is at a 30° angle for convenience





Removal Tool for Fixture Mount


- Removes the mount screw when a fixture and mount become wedged
- Connects to a driver handle and a torque wrench
- Insert vertically, and rotate it clock-wise to remove the mount
- App = Application



| App | Mini (TS,US) | Regular(TS,SS,US)/Wide(SS) | Wide (US) |
|-----|--------------|----------------------------|-----------|
| | ERFM | HRFR | ERFW |

Tissue Punch

- For flapless surgery
- Measures the height of gingiva, marked at 2mm increments
- Packing unit: tissue punch + guide pin
- ** Recommend using a tissue punch smaller than the healing abutment by 0.7 to 1.5mm.

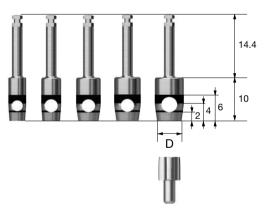

| D | Ø3.3 | ø3.8 | Ø4.3 | Ø4.8 | Ø5.3 |
|----|-----------|-----------|--------|--------|--------|
| | OSTP33 | OSTP38 | OSTP43 | OSTP48 | OSTP53 |
| | | | | | |
| TS | Ø 4.0/4.5 | Ø 4.5/5.0 | Ø 5.0 | Ø 6.0 | Ø 6.0 |
| SS | - | Ø 4.8 | - | ø 6.0 | Ø 6.0 |
| US | Ø 4.0 | Ø 5.0 | Ø 5.0 | Ø 6.0 | Ø 6.0 |
| | | | | | |

Based on application healing abutment

TS Bone Profiler

- Trims the bone surrounding a fixture for one stage and two stage procedures
- Connect the Guide Screw to the fixture in order to center the profiler. Make sure to compensate for the healing abutment.
- Guide Screw protects the fixture's platform from damage
- Packing unit : bone profiler + guide screw
- C = Connection



| C Healing Abutment D | Ø4.5 | Ø5.5 | Ø6.5/7.5 | |
|----------------------|--------|--------|----------|--|
| Mini/Regular | GSBP45 | GSBP55 | GSBP75 | |
| | | | | |

| Mini | Mini | Regular guide screw |
|---------------------|---------------------|---------------------|
| + | + | |
| Regular guide screw | Regular guide screw | |

US Bone Profiler


- Trims the bone surrounding a fixture and cover screw after a two stage procedure
- Remove cover screw, connect the Guide Screw to the fixture in order to center the profiler. Make sure to compensate for the healing abutment.
- Guide Screw protects the fixture's hex from damage
- Packing unit : bone profiler + guide screw
- P = Platform



| D \ P | Mini | Regular | Wide | T-type |
|-------|------------------|------------------|------------------|------------------|
| Ø4.0 | ABPM 400C | - | - | - |
| Ø5.0 | ABPM 500C | ABPR 500C | - | - |
| Ø6.0 | = | ABPR 600C | ABPW 600C | TBPW 600C |
| Ø7.0 | - | - | ABPW 700C | - |

Trephine Drill

- Harvests bone or removes a failed fixture
- Removes septal bone
- Also serves as the initial drill for ultra-wide fixture

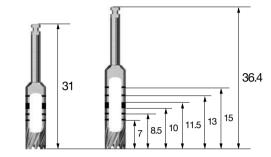

| L D (Inner / Outer) | 3.7/4.5 | 4.2/5.0 | 4.7/5.5 | 5.2/6.0 | 5.7/6.5 | 6.2/7.0 | |
|---------------------|---------|---------|---------|---------|---------|---------|--|
| Short | TD37S | TD42S | TD47S | TD52S | TD57S | TD62S | |
| Lona | TD37 | TD42 | TD47 | TD52 | TD57 | TD62 | |

Machine Driver Handle

• Manual handle for engine type surgical tools






Bone Mill


• Motar and pestle type to grind harvested autogenous bone into smaller particles.




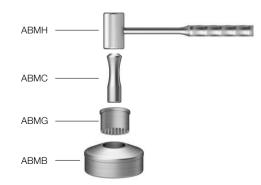


SmartBrush 1

- Cleans peri-implantitis
- Remove crown and abutment before use
- Recommended speed: 1,200 to 1,500rpm
- Recommended operation time: 2 to 3 min.
- Operating 5 min or longer can result in severe damage (fracture)
- Saline irrigation during the polishing process
- * Disposable; do not re-use; sterilized medical device (discard after use)

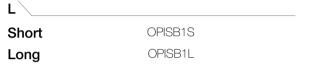


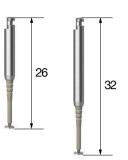

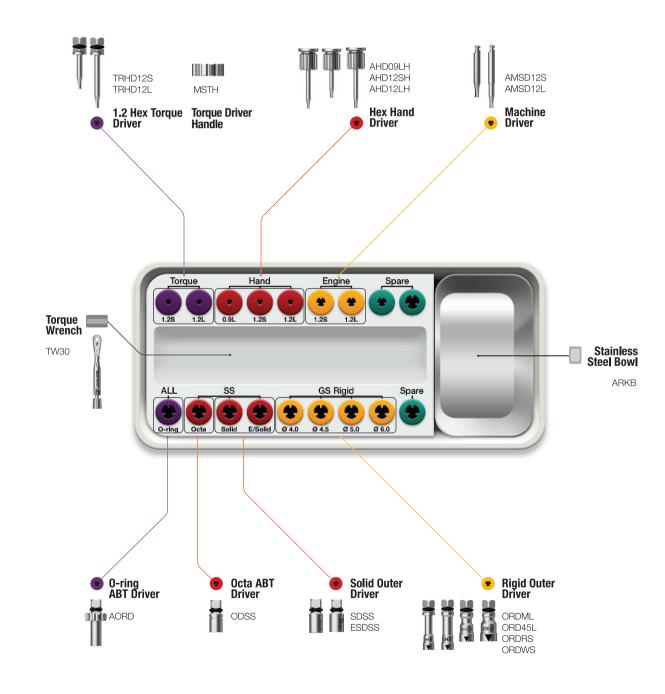


Prosthetic Instruments

Hand Driver

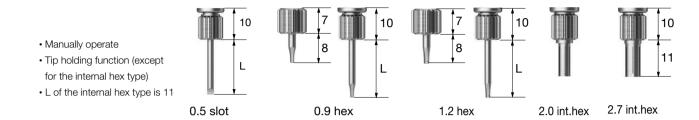

| L Type | 0.5 Slot | 0.9 Hex | 1.2 Hex | 2.0 Int.Hex | 2.7 Int.Hex |
|--------------|-----------------|------------------|-----------------|---------------|---------------|
| Ex.Short(8) | - | AHD 09MSH | AHD12MSH | - | - |
| Short (13) | ASD 05SH | AHD 09SH | AHD12SH | ⊩D 20H | ⊩D 27H |
| Middle (15) | = | = | AHD 12MH | = | = |
| Long (18) | ASD 05LH | AHD 09LH | AHD 12LH | = | - |
| Ex.Long (25) | - | - | AHD 12EH | - | - |

Machine Screw Driver

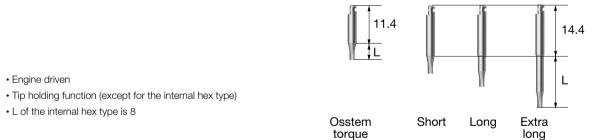

| L Type | 0.5 Slot | 0.9 Hex | 1.2 Hex | 2.0 Int.Hex | 2.7 Int.Hex |
|----------------|-----------------|-----------------|---------|----------------|----------------|
| Osstem Torque | (5) | - | OTH12S | - | - |
| Short (5.6) | AMSD 05S | AMSD 09S | AMSD12S | - | - |
| Long (11.6) | AMSD 05L | AMSD 09L | AMSD12L | EIHD 20 | EIHD 27 |
| Ex.Long (17.6) | - | - | AMSD12E | - | - |
| | | | | | |

Application


Driver connected products (Hand, machine screws, and torque drivers are all applicable)

Cover screw

Healing abutment, UCLA, screw, Mount screw

Esthetic abutment screw regular, Wide esthetic-low abutment screw

- Use with Torque Wrench
- Tip holding function
- Do not exceed the recommended torque value (over torquing may cause damage)
- Make sure fixture and diver is securly connected; loose connection may cause fixture fracture.
- When applying torque, keep the driver completely vertically (do not tilt)
- Replace the tip if there are any signs of wear and tear (bent, chipped, etc...)

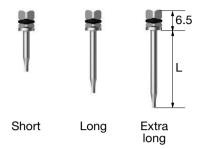


| L Type | 0.5 Slot | 0.9 Hex | 1.2 Hex | 2.0 Int.Hex | 2.7 Int.Hex |
|--------------|-----------------|-----------------|-----------------|-----------------|----------------|
| Ex.Short(8) | - | - | TRHD12MS | - | - |
| Short (13) | TRSD 05S | TRHD 09S | TRHD12S | TIHD 20S | - |
| Middle (15) | = | - | TRHD12M | - | - |
| Long (20) | TRSD 05L | TRHD 09L | TRHD 12L | TIHD 20L | TIHD 27 |
| Ex.Long (25) | TRSD 05E | - | TRHD12E | - | - |

052

Repair Torque Driver

- Handle size has been decreased compared to the torque driver (\emptyset 2.1 \to \emptyset 1.6)
- Minimizes the size of the crown hole during repair or SCRP procedure


| L Type | 1.2 Hex |
|------------|----------|
| Short (13) | TRHD12SR |
| Long (20) | TRHD12LR |

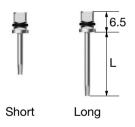

O-ring Abutment Driver

• Driver for o-ring abutment



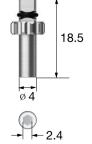

Rigid Outer Driver


- Driver for rigid abutment
- Recommended tightening torque: 30Ncm

| L \ Abutment D | Ø4.0 | Ø4.5 | Ø5.0 | Ø6.0 |
|----------------|-------|--------|-------|-------|
| Short (16.5) | ORDMS | ORD45S | ORDRS | ORDWS |
| Long (21.5) | ORDML | ORD45L | ORDRL | ORDWL |



Solid Abutment Driver

- Driver for solid abutment
- Before applying torque, connect and match the solid abutment groove with the driver's triangle marker
- Recommended tightening torque: 30Ncm

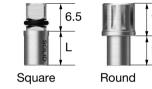

Regular

| Square | Round |
|--------|-------|
| SOLID | |
| SDSS | SDRS |
| SOLD | SOLD |
| SDSL | SDRL |
| | SDSS |

Wide



- Driver for excellent solid abutment
- Before applying torque, connect and match the solid abutment groove with the driver's triangle marker
- Recommended tightening torque: 30Ncm

Regular





Wide

| L Type | Square |
|------------|--------|
| Short (10) | ESD60S |
| | |

Octa Abutment Driver


- Driver for octa abutment
- Recommended tightening torque: 30Ncm

| L Type | Square | Round |
|--------|--------|-------|
| Short | ODSS | ODRS |
| Long | ODSL | ODRL |



Osstem Torque Driver

- Driver exclusively for the Osstem torque. Do not use with other general hand pieces
- Match the driver's triangle marker with the groove or cross-section of an abutment
- ullet Solid and excellent solid drivers are only compatible with the $^{\emptyset}$ 4.8
- 1.2 hex type L is 5

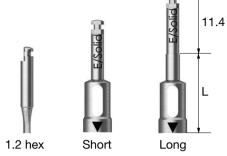


| L Type | 1.2 Hex | Rigid 4.0 | Rigid 4.5 | Rigid 5.0 | Rigid 6.0 | Solid | Excellent Solid | |
|------------|---------|-----------|-----------|-----------|-----------|--------|-----------------|--|
| Short (10) | OTH12S | OTR40S | OTR45S | OTR50S | OTR60S | OTS48S | OTE48S | |
| Long (15) | - | OTR40L | OTR45L | OTR50L | OTR60L | OTS48L | OTE48L | |

Path Probe for TS

- \bullet Checks the implant's path and measures the height of the gingiva after placing a TS fixture
- C = Connection

| <u> </u> | Mini | Regular | |
|----------|-------------|-------------|--|
| | GIPAP-3016A | GIPAP-3516A | |



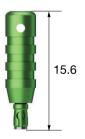

Torque Connector

• Connects the torque square driver to the bi-directional torque wrench





05

Machine Driver Connector

• Connects the machine square driver to the bi-directional torque wrench

OMDC



Driver Handle

• Connects to the torque driver


TIDHC



Dalbo Plus Screw Driver

• Adjusts the retention force of the Dalbo plus attachment

ODSD

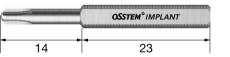

Finishing Reamer Set

• Removes the casting body's internal lip of the plastic coping





Reamer user guide

- 1. Select a reamer tip that is the same size as the
- abutment, and connect it to the burn-out cylinder
 2. Firmly grasp the casting body and rotate the
 Reamer Bite with consistent force
- 3. Ream the body until the it is clean and free of the





• Cutting blade removes the casting body's internal lip of the plastic coping


FRBC



Reamer Tip for Rigid Abutment


• Guides the reamer when removing the casting body's internal lip of the plastic coping

\ **D** Ø4.0 Ø4.5 Ø5.0 Ø6.0 GSRFRT400 GSRFRT450 GSRFRT500 GSRFRT600

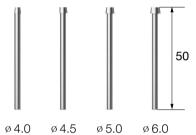

- Guides the reamer when removing the casting body's internal lip of the plastic coping
- The guide for $\,^{\varnothing}$ 6.0 Solid and excellent $\,^{\varnothing}$ 4.8 Solid are interchangeable

| D | Ø4.8 | Ø6.0 | |
|----------|---------|---------|--|
| Solid | FRTS480 | FRTS600 | |
| Ex.Solid | FRTE480 | FRTE600 | |

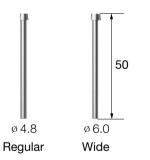


20.9

20.9

36.9

059

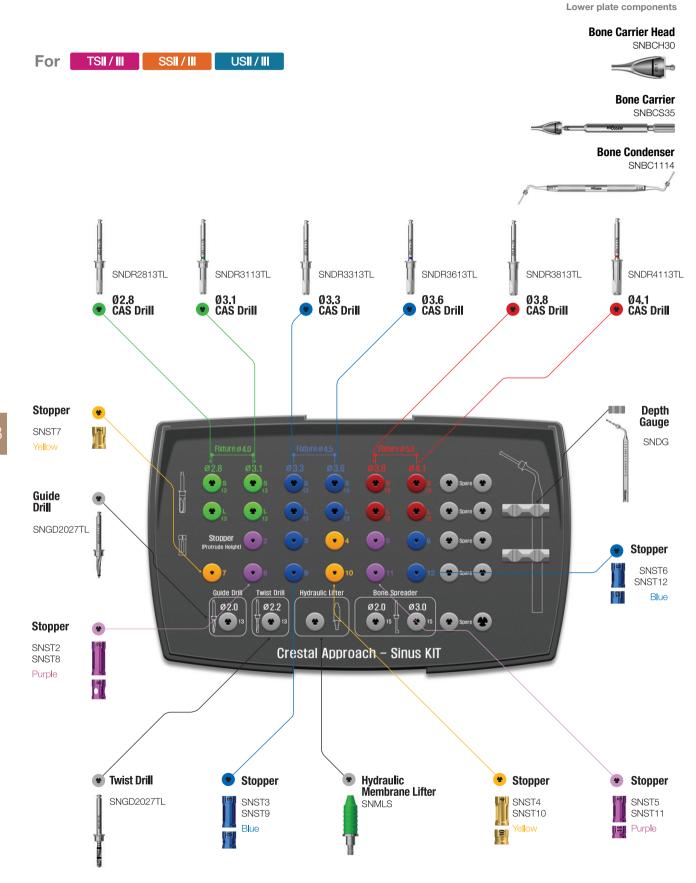

CAS KIT Surgical Instruments

CAS Drill

- Specallized drill designed to penetrate the sinus floor without damaging the Schneiderian membrane by forming a concial bone lid.
- Four blade body drills well at both high and low speeds and is capable of collecting autogenous bone at low speeds.
- Use with stoppers for safe and controlled penetration
- Final drill should be based on the bone quality, regardless of the fixture type (straight or tapered).

Ø2.8 Ø3.1 Ø3.3 Ø3.6 Ø3.8 \ D Ø4.1

• Recommended speed: 400~800rpm (First time: 400rpm) 16 SNDR2813TL SNDR3113TL SNDR3313TL SNDR3613TL SNDR3813TL SNDR4113TL

Guide Drill

- Marks the fixture's insertion site
- · Side cutting blades trims the extraction socket sidewalls
- Marker 2mm from the tip

D Ø2.0/2.7 SNGD2027TL

Ø2.2 Twist Drill


- Recommend under-drilling by 1mm less than the bone's thickness
- · Use with stoppers for safe and controlled drilling
- The tip measures an additional 0.6 mm


\ D Ø2.2 SNTD2213TL

Hydraulic Membrane Lifter Set

- Hydraulic pressure is used to separate and lift the sinus membrane
- Securly fits $\,^{\varnothing}$ 2.8 to $\,^{\varnothing}$ 4.1 CAS drilled osteotomies

Ø2.6/6.0 D SNMLS

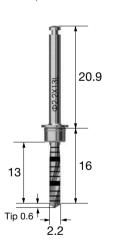

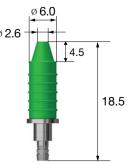

CAS KIT Surgical Instruments

Stopper

- Laser marked numbers indicate the remaining tool's (drill, instruments, etc.) length
- · Color-coded by length
- Drill and Stopper recommended number of usage is 50 times




Bone Carrier

- Handle for the Bone Carrier Head
- Connect the Bone Carrier Head and tighten at the opposite end.
- Connects both heads (SNBCH30 or SNBCH35)



Bone Carrier Head

- Cone shaped with an extended tip that reaches the sinus cavity and prevents bone material from spilling out.
- SNBCH30 for Ø 3.1/ Ø 3.3 CAS drilled osteotomy
- SNBCH35 for Ø 3.6/ Ø 3.8/ Ø 4.1 CAS drilled osteotomy
- Fill the reservior with bone material (up to the marker), with the Bone Condenser shuttle the material in small quantities into the sinus. Repeat the process as necessary.

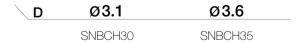

Bone Condenser

• Safely shuttles bone material through the Bone Carrier Head into the sinus cavity. - SNBCH30: Use Ø 1.1 / SNBCH35: Use Ø 1.4







Hydraulic Membrane Lifter Tube

• Tubing connects to the Hydraulic Membrane Lifter and salin filled syringe.

SNMT



Depth Gauge

• Measures the thickness of the residual bone and checks to see if the sinus is properly separated from the floor.






CAS KIT Plus (HCRSNKP)

Incorporates six short CAS Drills

CAS Drill

- Specailized drill designed to penetrate the sinus floor without damaging the Schneiderian membrane by forming a concial bone lid.
- Four blade body drills well at both high and low speeds and is capable of collecting autogenous bone at low speeds.
- Stopper system for safe and controlled penetration of the sinus floor without membrane damage.
- Final drill should be based on the bone quality, regardless of the fixture type (straight or tapered).

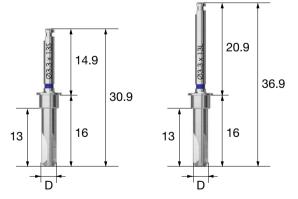

| L \ D | Ø2.8 | Ø3.1 | Ø3.3 | Ø3.6 | Ø3.8 | Ø4.1 | |
|-------|------------|------------|------------|------------|------------|------------|--|
| Short | SNDR2813TS | SNDR3113TS | SNDR3313TS | SNDR3613TS | SNDR3813TS | SNDR4113TS | |
| Long | SNDR2813TL | SNDR3113TL | SNDR3313TL | SNDR3613TL | SNDR3813TL | SNDR4113TL | |

Dome Drill

- Forms a bone window, at the same time collects autogenous bone
- Excellent penetration due to the Macro and micro cutting blades
- Stopper safely controls the penetration depth
- Recommended speed: 1,200 to 1,500 rpm
- * Excessive drilling may cause damage to the membrane.

| $L \setminus D$ | Ø5.5 | Ø7.0 | Wide Ø7.0 |
|-----------------|-----------|----------|-----------|
| 25 | LSDR554TD | LSDR74TD | LSDR74WTD |





Core Drill

- Forming a bone window and cores a bone lid
- Excellent cutting ability and based on the CAS drill design there is no membrane damage
- Recommended speed: 1,200 to 1,500rpm
- * Excessive drilling may cause damage to the membrane

| L \ D | Ø5.5 | Ø7.0 |
|-------|-----------|----------|
| 25 | LSDR554TC | LSDR74TC |



Side Wall Drill

- Enlarges the bone window after using the dome drill
- Cut using the blade 1mm above the bottom of the drill
- Recommended speed: 1,500rpm

| SI | NDR36T | | | | | |
|--------------------------------------|--------|-----------------------|-----|--------|-----|---|
| Height of side cutting blade (mm) | 1.0 | 2.0 | 3.0 | 4.0 | 5.0 | |
| CAS KIT stopper(mm) | 8.0 | 9.0 | 10 | 11 | 12 | - |
| Side wall drill + CAS KIT stopper | | ၂ ၈) [(| 9 | , I=I, | | н |

* Stopper safely controls the penetration depth





HST75


Stopper

- Laser marked numbers indicate the remaining tool's (drill, instruments, etc.) length
- Color-coded by length
- Drill and Stopper recommended number of usage is 50 times


| \ L | 0.5 | 1.0 | 1.5 | 2.0 | 2.5 | 3.0 |
|-------|----------|----------|----------|----------|----------|----------|
| | | | | | | |
| | · | 15 | 5 | 20 | 22.5 | 30 |
| | LSNST0.5 | LSNST1.0 | LSNST1.5 | LSNST2.0 | LSNST2.5 | LSNST3.0 |
| Color | Yellow | Purple | Blue | Yellow | Purple | Blue |


LAS KIT Plus (HLRSNKP)

• Incorporates six additional sinus lifting tools

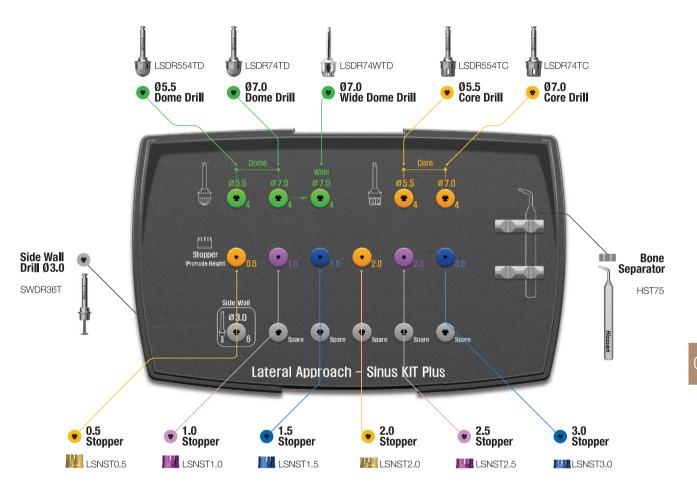

LAS KIT Plus Lower Plate

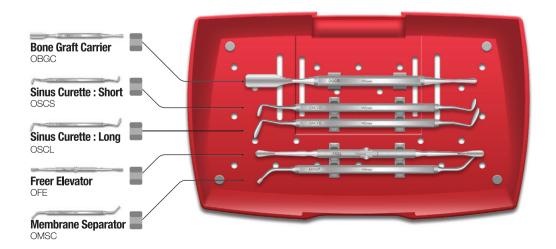

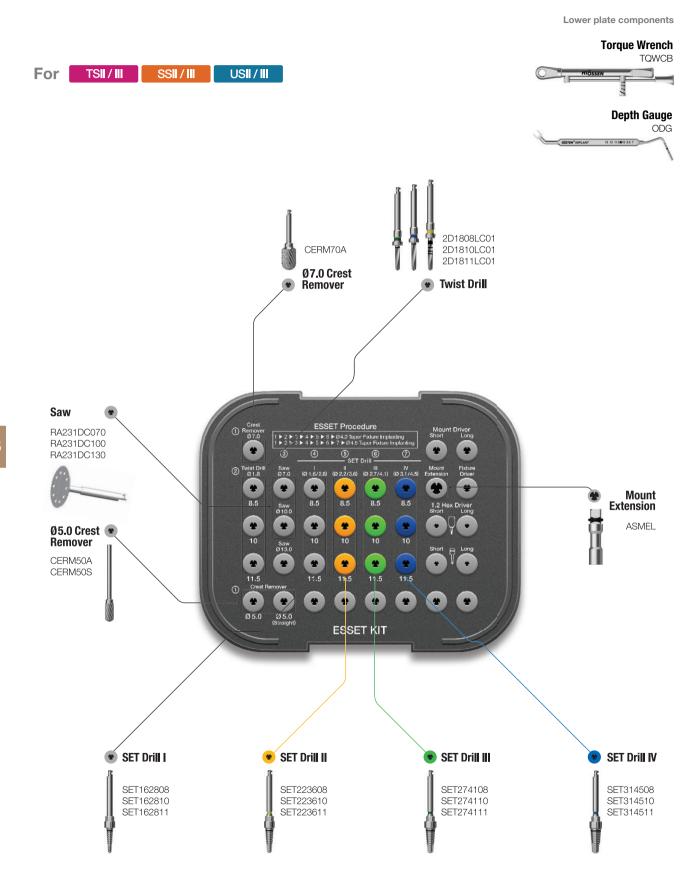


ESSET KIT Surgical Instruments

Crest Remover

- Grinds down narrow aveolar ridge, and creates an indentation for the fixture's insertion site
- Angled Type recommended speed: 1,200 to 1,500 rpm
- Straight Typ recommended speed: 15,000 to 30,000 rpm


| L \ D | Ø5.0 | Ø7.0 |
|-------|---------|---------|
| 29 | CERM50A | CERM70A |
| 45 | CERM50S | = |

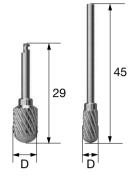

Twist Drill

- Marks the fixture's insertion site
- Slide on the Stopper to control the depth
- Recommended speed: 1,200 to 1,500rpm

| L TL D | Ø1.8 | |
|---------|------------|--|
| 8.5 33 | 2D1808LC01 | |
| 10 34.5 | 2D1810LC01 | |
| 11 36 | 2D1811LC01 | |

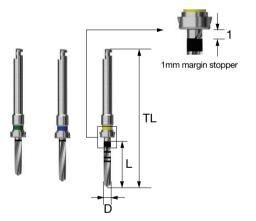


067

Saw

- Saws narrow aveolar ridge
- Saw vertically first, then saw from the mesial to the distal
- Recommended speed: 1,200 to 1,500rpm
- T = Thickness


| T \ | Ø7.0 | Ø10.0 | Ø13.0 |
|-----|------------|------------|------------|
| 0.3 | RA231DC070 | RA231DC100 | RA231DC130 |

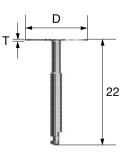

SET Drill

- Expands narrow aveolar ridge
- Use the SET drills in numerical order based on the diameter of the fixture F4.0: I \to II \to III / F4.5: I \to II \to IV
- Recommended speed: 25 to 35rpm

| L | Туре | I | II | Ш | IV |
|------|-------|-------------------|-------------------|-------------------|-------------------|
| | D1/D2 | Ø1.6/2.8 | Ø2.2/3.6 | Ø2.7/4.1 | Ø3.1/4.5 |
| 8.5 | | SET 162808 | SET 223608 | SET 274108 | SET 314508 |
| 10 | | SET162810 | SET 223610 | SET 274110 | SET 314510 |
| 11.5 | 5 | SET 162811 | SET 223611 | SET 274111 | SET 314511 |

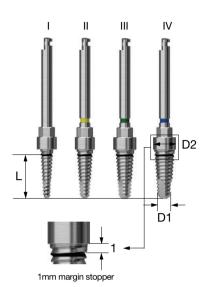

Mount Extension

• Connect with SET Drills for manual torque

ASMEL



Torque Wrench

• Use with Mount Extension and SET Drills





Depth Gauge

• Releases a wedged SET Drill due to over torquing and fir when the hand piece ceases because bit is stuck. Use with an open wrench.





Saw Protector

- Saw cover prevents debris from ejecting outside the oral cavity and protects adjent soft tissue
- Cover can rotate 360° adding convenience during surgery.
- Contra angle type: Detachable saw cover -

KaVo (CL 3-09, S201L), W&H (WS-75)

- Straight type: Integrated saw cover KaVo (CL10)
- * Use an appropriate saw
- X Cover and body need to be ordered separately



Contra angle type




Straight type

| Type | D | | Co | Cover | | |
|------|-------------------|---------|---------|---------|--------|-------|
| | | Ø7.0 | Ø10.0 | Ø13.0 | Ø15.0 | |
| Kavo | Contra Angle Type | SP07AC | SP10AC | SP13AC | - | SPAB |
| | Straight Type | - | SP10SC | SP13SC | SP15SC | - |
| W&H | Contra Angle Type | SP07ACW | SP10ACW | SP13ACW | - | SPABW |


06

MS KIT (OMSK)

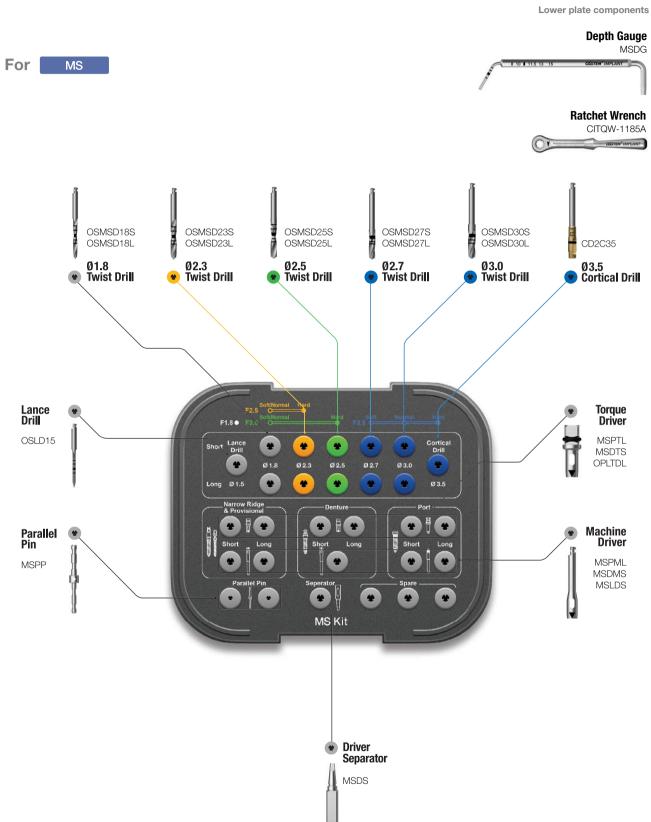


Ortho KIT (OOKS)

For OS

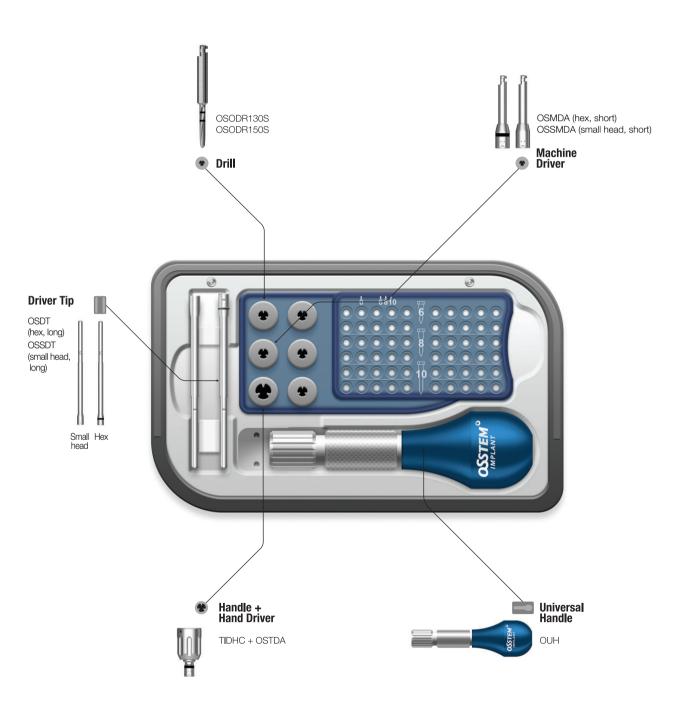


Bone Screw KIT (BSSTKT)

For BS

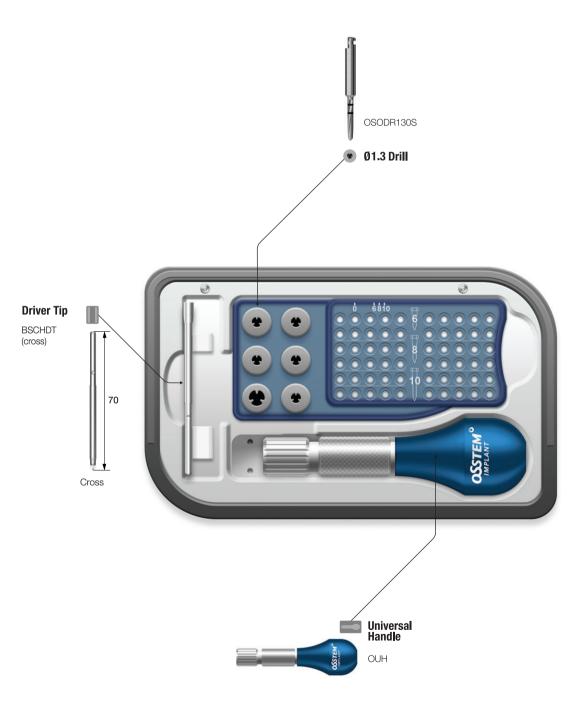


Custom KIT (OCTK)

- Sterilizable case for storing extra tools
- Includes three types of rubber (large, medium, and small) holders
- Sterilization parameters (132℃, 15min)

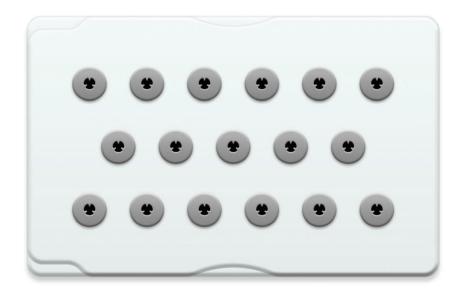


Osteo KIT (OSTK)

- Crestal approach sinus lift surgery
- Osteotome is designed to compact bone while pentrating the sinus floor.
- Includes stopper system for safe and controlled penetration.

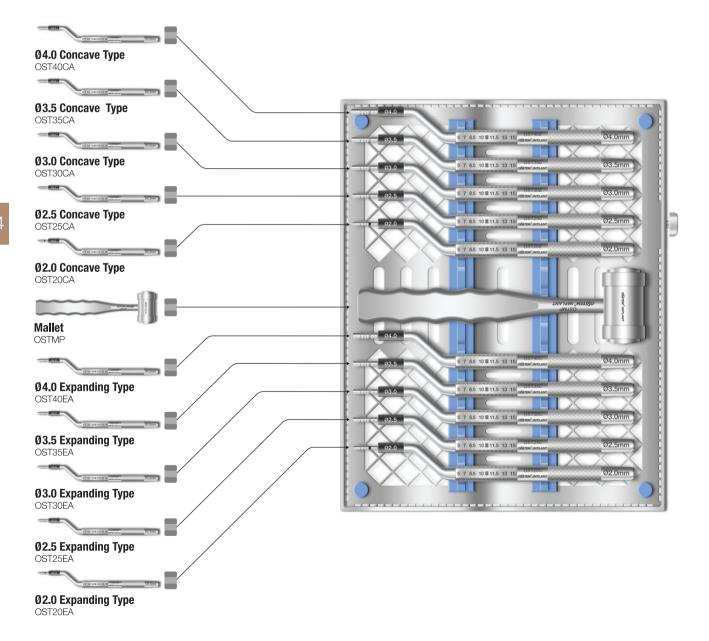


Osteotome KIT (AOST)

- Crestal approach sinus lift surgery
- Concave type only
- Includes stopper system for safe and controlled penetration.

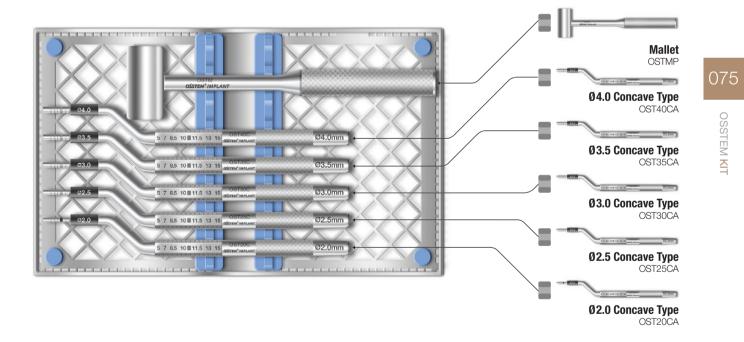


Sinus KIT (ASLK)


- Tools for lateral approach sinus floor elevation surgery
- Components (5 types)
- Freer elevator : OFE
- Bone graft carrier : OBGC
- Membrane separator (circle type) : OMSC
- Sinus currette-short : OSCS
- Sinus currette-long : OSCL

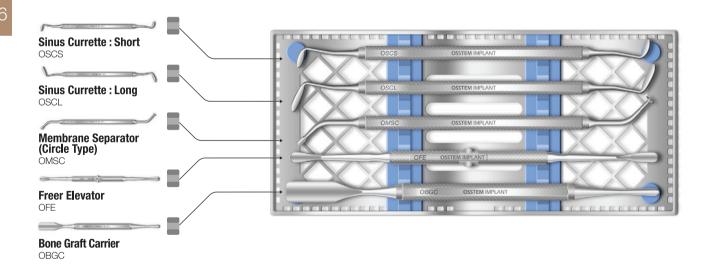

Bone Spreader KIT (OBSOK)

- Expands narrow alveolar ridge
- Offset type
- · Components (4 types)
- OBSO22F, OBSO28F, OBSO35F, OBSO35R

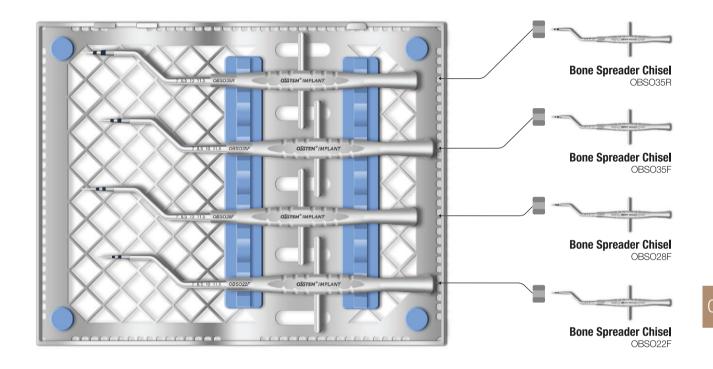

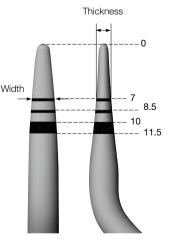


- Use for alveolar bone expansion
- Offset type for easy operation
- Depth marking corresponding to the implant length.


| Malletting |
|---|
| Direction for use: refer to the above schematic |

| | | | | | (Unit:mm) |
|--------------|------------------|------|------|------|-----------|
| Code | Tip length Spec. | 7 | 8.5 | 10 | 11.5 |
| 000000 | Thickness | 1.15 | 1.3 | 1.45 | 1.6 |
| OBSO22F | Width | 2.1 | 2.2 | 2.2 | 2.2 |
| OBSO28F | Thickness | 1.15 | 1.3 | 1.45 | 1.6 |
| OB3026F | Width | 2.65 | 2.8 | 2.8 | 2.8 |
| 0000055 | Thickness | 1.3 | 1.45 | 1.6 | 1.8 |
| OBSO35F | Width | 3.3 | 3.5 | 3.5 | 3.5 |
| OBSO35R | Thickness | 1.85 | 2.1 | 2.3 | 2.55 |
| (round type) | Width | 3.3 | 3.5 | 3.5 | 3.5 |

Ridge Split KIT - Straight (ORSSK)


- Chisel: Expands narrow alveolar ridge
- Blade holder: Cuts poor bone quality using a bur, malletting is possible, use a #15 blade
- Ridge split chisel: ORSS15, ORSS20, ORSS25, ORSS30
- Blade holder : ORSBH

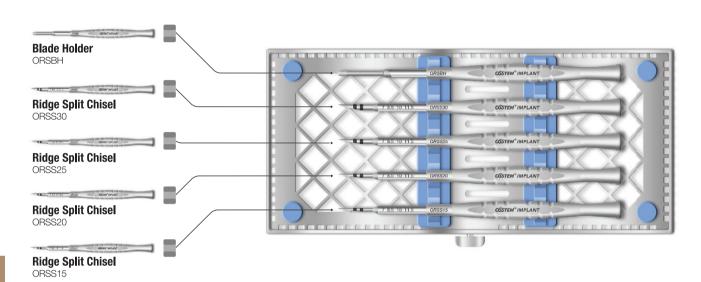


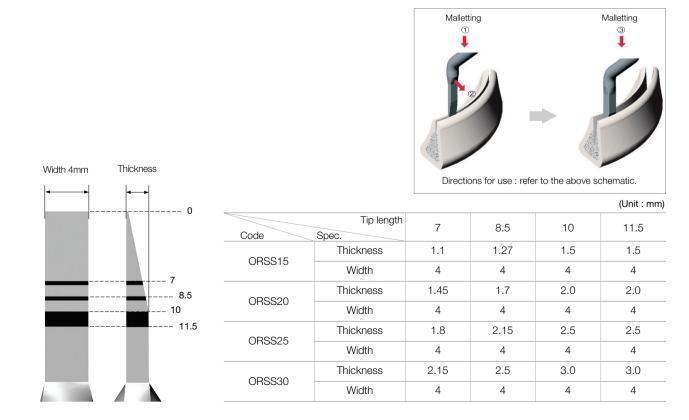


Ridge Split KIT - Offset (ORSOK)

- Chisel: Expands narrow alveolar ridge
- Blade holder: Cuts poor bone quality using a bur, malletting is possible, use a #15 blade
- Ridge split chisel: ORSO15, ORSO20, ORSO25, ORSO30
- Blade holder : ORSBH

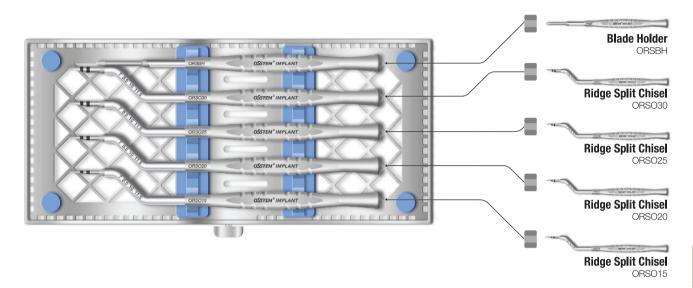

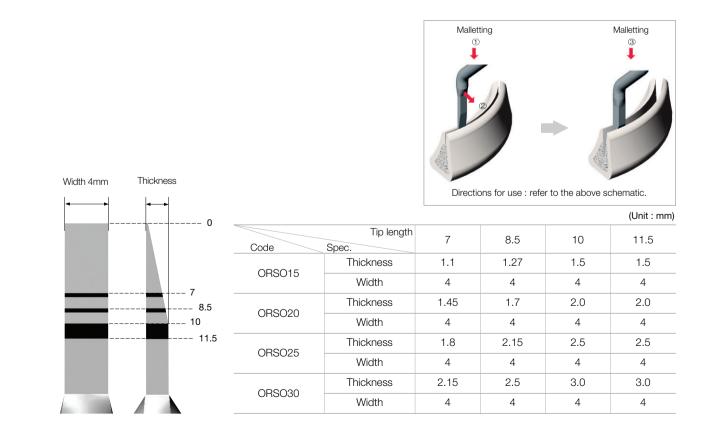

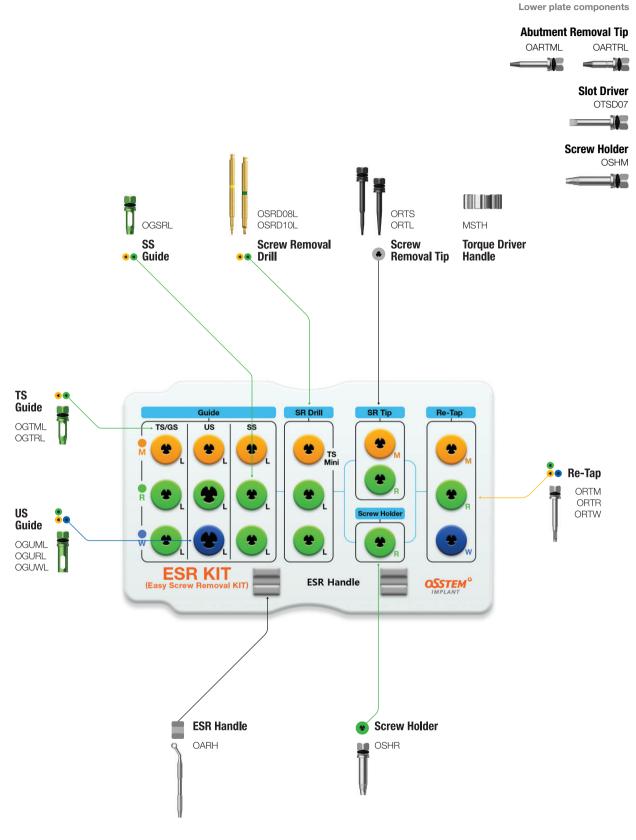

ESR KIT Surgical Instruments

Reverse Driver

- · Removes fractured screws
- Select the propriate Guide that matches the fixture
- Operate the driver in reverse, when the red marker appears above the Guide, stop and disconnect the driver. Connect the Screw Holder to remove the screw.
- For hand mode / Rotate counterclock wise / Number of usages: 10 times
- F = Fixture

| L \ F | Mini | Regular/Wide |
|-------|--------|--------------|
| Short | - | ORVDRS |
| Long | ORVDML | ORVDRL |

Regular/Wide

Screw Removal Drill (SR Drill)

- Drills a hole in a factured screw
- Make sure to connect the guide before use; irrigate generously with saline solution and suction out any debris
- Available in Long and Short lengths for different intermaxillary distances
- Drill until the color marker is no longer visible
- Recommended speed: 1,200 to 1,500 rpm (counterclock wise)
- Connect the guide before use/Do not apply excessive vertical force/Do not clean with hydrogen peroxide.
- * Disposable; do not re-use
- Short: Single unit purchase available.
- F = Fixture


| L\F | Mini | Regular/Wide(GS/TS/SS/US) | | |
|-------|---------|---------------------------|--|--|
| Short | OSRD08S | OSRD10S | | |
| Long | OSRD08L | OSRD10L | | |



Guide

- Centers and guides Reverse Driver, Screw Removal Drill (SR drill), Re-tap, and minimizes vibration
- Available in Long and Short lengths for different intermaxillary distances
- Connect to ESR Handle before use
- Short: Single unit purchase available
- F = Fixture

| F Type | TS(I | Hex) | SS | Octa) | US | Hex) |
|---------|-------|-------|-------|-------|-------|-------|
| | Short | Long | Short | Long | Short | Long |
| Mini | OGTMS | OGTML | OGUMS | OGUML | OGUMS | OGUML |
| Regular | OGTRS | OGTRL | OGSRS | OGSRL | OGURS | OGURL |
| Wide | - | - | OGSRS | OGSRL | OGUWS | OGUWL |

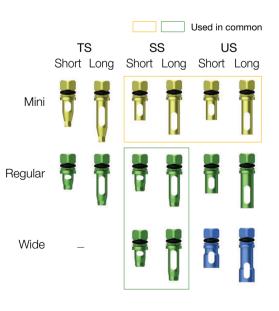

Screw Removal Tip (SR Tip)

- Engage counterclock wise into the drilled hole made by the Screw Removal Drill (SR Drill) of a factured screw, continue to rotate to remove screw.
- Rotation direction: Counterclock wise
- Disposable; do not re-use.

| L | Mini | Regular/Wide |
|-------|--------|--------------|
| Short | ORT16S | ORTS |
| Long | ORT16L | ORTL |



Screw Holder

- Grasps onto a protruding fracture screw and unscrews it
- Color-coded
- F = Fixture

| \ F | Mini | Regular | Wide |
|-----|------|---------|------|
| | OSHM | OSHR | OSHW |



Re-tap

- Re-threads the internal connection of a fixture, if the screw does not properly engage and tightens.
- Connects to a torque wrench or ratchet wrench to re-thread
- F = Fixture

| F | Mini (M1.6) | Regular (M2.0) | Wide (M2.5) |
|---|-------------|----------------|-------------|
| | ORTM | ORTR | ORTW |



• Centers and guides tools into the middle of the fixture

OARH

Abutment Removal Tip (AR Tip)

- Removes the remaining part of a factured abutment or mount still in a fixture.
- Engage the tip into the fractured abutment counterclock wise. Using forceps grasp the Removal Tip and rock back and forth until the factured abutment is freed.
- The tip can be used remove a screw with a stripped hex
- To remove the screw, engage the tip into the stripped hex and rotate counterclock wise.
- F = Fixture

| \ F | Mini | Regular |
|---------|---------|---------|
| Short | OARTMS | OARTRS |
| Long | OARTML | OARTRL |
| Ex.Long | OARTMEL | OARTREL |

Transfer Abutment Separate Tool

- Removes stuck or wedged non-hex transfer abutments
- Separate Tool tip fits mini abutments; regular tools can also be used through the second groove.
- After removing the abutment screw, insert the Separate Tool body into the abutment. Fasten the driver, securely joining the Separate Tool body and abutment. Remove the abutment. If this does not release abutment from the fixture, retighten with a ratchet wrench to the driver and try again.

| Driver | Body | Set |
|--------|------|------|
| TASD | TASB | TAST |

Slot Driver

• Cut a slot on a stripped hex healing abutment, cover screw, or abutment screw using a Ø0.8 bur to unscrew.

OTSD07



Body



Driver

Fracture mode

OTE

EFR KIT Surgical Instruments

Remover Screw

- Connects to the failed implant and serves to support the Remover Body
- Available in different sizes to match the diameter of the fixture to be removed (TS/SS/US, normal/fracture)
- Fracture type is specifically for removing a fractured fixture
- Recommended tightening torque: regular/wide 100Ncm, mini 80Ncm
- * Disposable; do not re-use.
- F = Fixture
- P = Platform

| | Mini | Regular | Wide |
|--------------|----------|---------------|-----------|
| Type F/P | Ø3.5/- | Ø4.0~4.5/P4.8 | Ø5.0/P6.0 |
| TS/SS Normal | FRSM35 | FRSR40 | FRSW50 |
| Fracture | FRSM35F | FRSR40F | FRSW50F |
| US | FRSM35US | FRSR40US | FRSW50US |

Screw Driver

- Connects and fastens the Remover Screw to the fixture
- Recommended tightening torque: regular/wide 100Ncm, mini 80Ncm

| F | Mini | Regular | Wide | |
|---|---------|---------|---------|--|
| | FRSDM23 | FRSDR25 | FRSDW30 | |

Remover Body

- Connects to a failed fixture via the Remover Screw and by applying counterclock wise torque, removes the implant.
- Available in different sizes to match the diameters of the fixture to be removed.
- * Disposable; do not re-use.
- F = Fixture

| F | Mini | Regular | Wide |
|----------|---------|---------|---------|
| Short | FRBM35S | FRBR40S | FRBW50S |
| Long | FRBM35L | FRBR40L | FRBW50L |







• Connect with Screw Driver to fasten and Remover Body to remove the fixture

OTE

- Applies up to 400Ncm of torque (markers at 80/100/200/300/400Ncm).
- Torque by pulling the bar back until reaching the desired torque value.
- · Clean and sterilize for storage.

TW400B



• Removes implants from the Remover Body after removing the fixture from the bone

FRDFE



Painless Anesthesia System

SlowJec Anesthesia System -Wireless & Painless anesthesia system

- Slowly injects anesthia, minimizes pain and pressure to the injection site
- The system monitors soft tissue pressure and automatically stops injection if it surpases a certain limit. Once the pressure disapates, the remaining dose is automatically delivered.
- No hand fatigue and more control over anesthetic usage
- No cords to get in the way; voice prompts sooth patients

SLOWJEC




Parallel Guide KIT

Prime KIT: OPGPK (optional tools are not included)
Advanced KIT: OPGAK (optional tools are all included)

L-Wrench Option
LWC20H

Denture Guide Option
PGODA

Multi Joint Handle Option
MJH

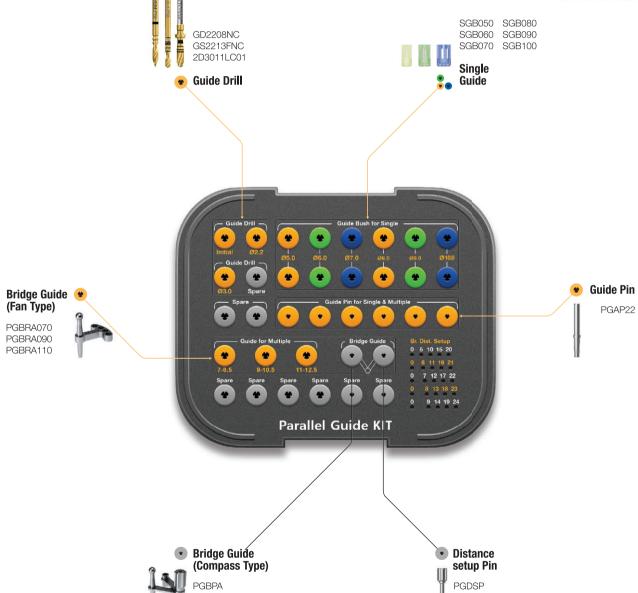


Parallel Guide KIT Surgical Instruments

Guide Drill


- Initial drill: Drilling depth can be adjusted by fastening it to the Single Guide
- Ø 2.2 twist drill: Used with the Bridge Guide
- Ø 3.0 twist drill: Final drill

| <u>D</u> | Ø2.2 | Ø3.0 |
|---------------|-----------|------------|
| Initial drill | GD2208NC | - |
| Twist drill | GS2213FNC | 2D3011LC01 |

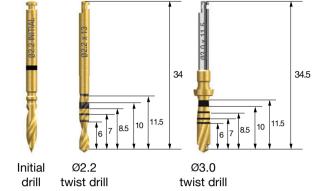

Single Guide

- Transparent material indicates the location and direction of drilling
- Available in six sizes from Ø 5.0 to Ø 10.0, must take into account the mesiodistal crown diameters
- Packing unit: 2EA
- * Disposable; do not re-use
- ** Drilling depth can be adjusted from 6 to 8mm, refer to the initial drill marker and top of the Single Guide marker.

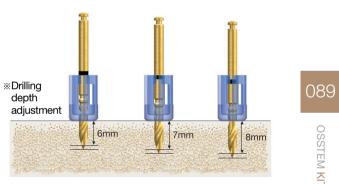



Guide Pin

• Checks drilling path and secures the Single Guide





Bridge Guide

- Adjustable drill guide for setting up the optimal implant placement and initial drilling sites.
- Fan type: Range between 7.0 mm to 12.5 mm, 0.5mm increments
- Compass type: Range between 5.0 mm to 24 mm, 1mm increments
- Set distance using the kit's middle plate





| Type Distance | 7~8.5 | 9~10.5 | 11~12.5 | 5~24 |
|---------------|----------|----------|----------|-------|
| Fan | PGBRA070 | PGBRA090 | PGBRA110 | - |
| Compass | = | = | = | PGBPA |

90

Multi Joint Handle Option

• Handle connects to the ball head of the Bridge Guide, provides information about the guide from outside the mouth





Denture Guide Option

- Angle adjustable denture guide for fully edentulous cases
- Using a stone model, arrange the guide to the ideal confirguration.
 Tighten and set the guide using the L-wrench. Transfer to the patient to start surgery.
- Markers represent tooth positions, 2, 3, 4, 5, etc... starting from the midline





L-wrench Option

• Tightens the denture guide after size adjustments

LWC20H

Distance Setup Pin Option

• Compass type bridge guide and Pin type denture guide

PGDSP



OSSTEM KIT

Guide Pin (4ea) SGP 22 Round bur (2ea) RAHM1018 **Ø2.2 Cast Drill (2ea)** For stone models 2D2208LC01 SGCB30S Checkup Ø2.2 Twist Drill Ø3.0 Twist Drill Smart SGTD2207S ø2.2 Drill QGTD3008 Smart Template **SmartGuide KIT** 2-unit Br.: small Single Free-end Bridge 2-unit Br.: large SGTSS SGTFB90LS SGTB63SS SGTB85LS

SmartGuide KIT Surgical Instruments

Smart Template

Lower plate components

- Medical grade thermoplastic material
- \bullet Becomes flexible when immersed in 70°c water for approx. 1 min.
- Template hardens in 1 min. at room temperature
- * Disposable; do not re-use; sterilizable under low temperature (Do not autoclave, do not use hydrogen peroxide)

| Туре | Single | Free-end Bridge | 2-Unit Br.: small | 2-Unit Br.: large |
|------|--------|-----------------|-------------------|-------------------|
| | 0 | 00 | 00 | 00 |
| | SGTSS | SGTFB90LS | SGTB63SS | SGTB85LS |

Twist Drill

- Drills specifically for Smart Template
- Stable drilling through the Smart Template sleeve
- Initial drilling using the $\,^{\varnothing}$ 2.2, followed by $\,^{\varnothing}$ 3.0 drill
- Recommended speed: 1,200 to 1,500rpm

| <u>D</u> | Ø2.2 | Ø3.0 | |
|----------|-----------|----------|--|
| | SGTD2207S | QGTD3008 | |



Checkup Bush

- Connect to the Smart Template sleeve and insert into the surgical site.
- \bullet Radiographically (CT or Xray) check the drilling positions





SmartGuide KIT Surgical Instruments

Round Bur

- Marks site of the Guide Pin on a stone model
- Number of usages: 10 times
- Recommended speed: 1,200 to 1,500rpm

\ D

Ø1.8

RAHM1018

Ø2.2 Cast Drill For stone models

- Drills the hole in the stone model for the Guide Pin
- Number of usages: 10 times
- Drill after marketing the site with the round bur
- Recommended speed: 1,200 to 1,500rpm

 $\setminus D$

Ø2.2

2D2208LC01

Guide Pin

- Pin that secures the Smart Temple to the stone model
- Connected to the smart template sleeve


SGP22











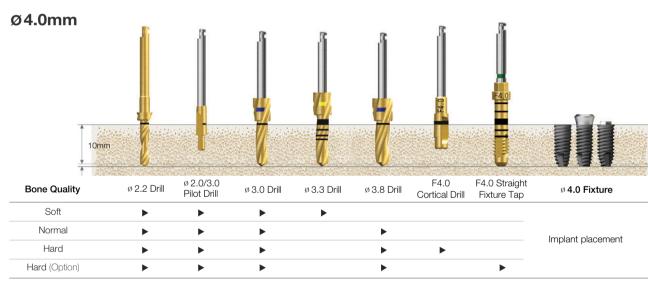

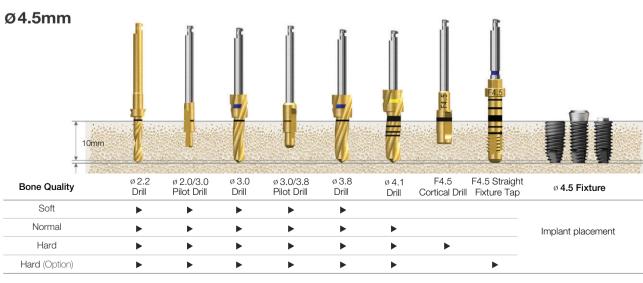

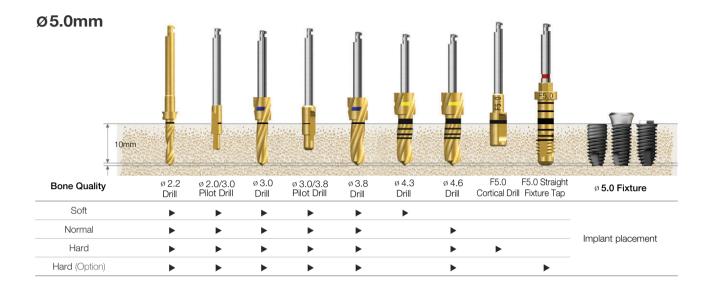



TS fixture insertion depth: In normal/hard bone should be inserted 1mm subcrestal. In soft bone, it should be inserted at bone level for higher stability. The fixture tap should used for hard bone.


Recommended speed is 25rpm or hand torgue using a mount extension and torque wrench.


TSIII CA/SA/BA/HA | SSIII CA/SA/HA | USIII CA/SA

(Length: 10mm)

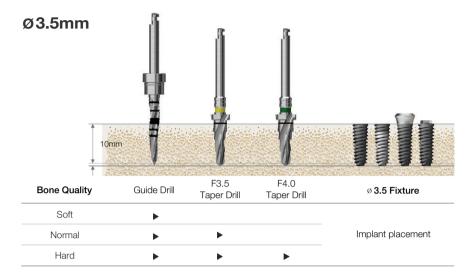

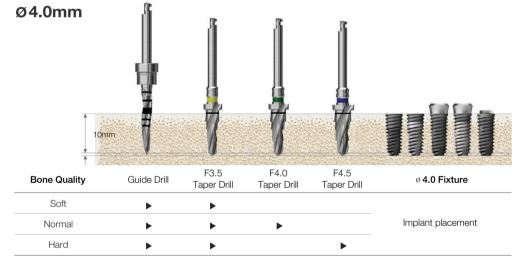

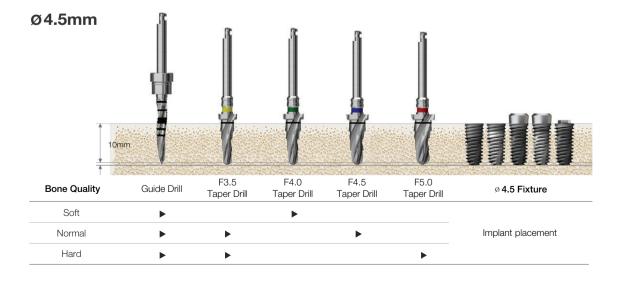


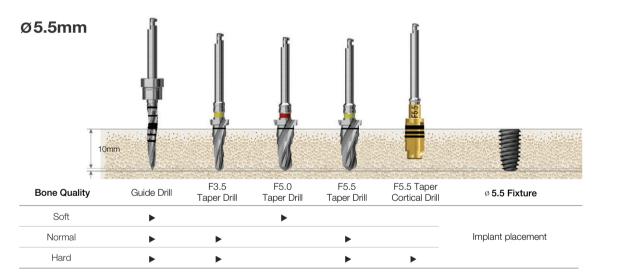



 $\textbf{F5.0 taper cortical drill marker: } \ Drill \ to \ the \ bottom \ of \ the \ marker \ for \ fixture \ lengths \ \geq 6 mm, \ the \ middle \ for \ fixture \ lengths \ \geq 7 mm, \ and \ the \ top \ for \ \geq 8.5 mm$ Recommended insertion torque: ≤40Ncm, for the TSIII/SSIII HA: ≤35Ncm (the HA coating can fracture and flake off when placed in hard bone) TS fixture insertion depth: In normal/hard bone should be inserted 1mm subcrestal. In soft bone, it should be inserted at bone level for higher stability.

TSIII CA/SA/BA/HA | SSIII CA/SA/HA | USIII CA/SA

(Length: 10mm)

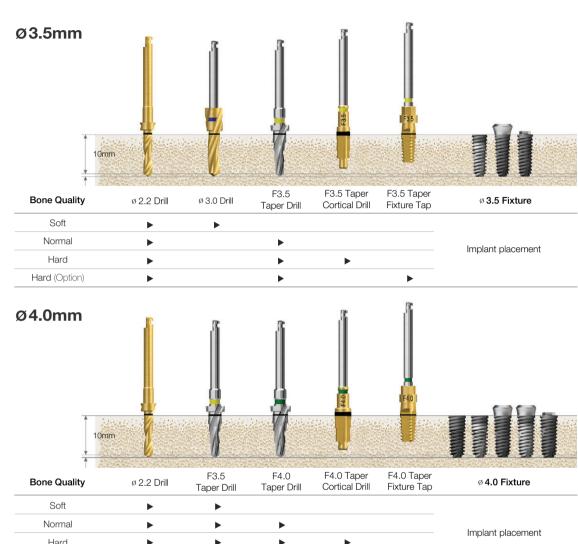

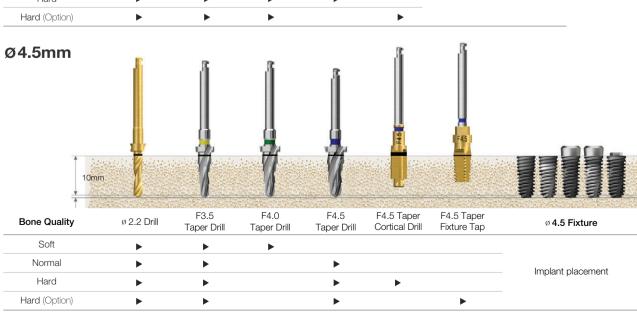

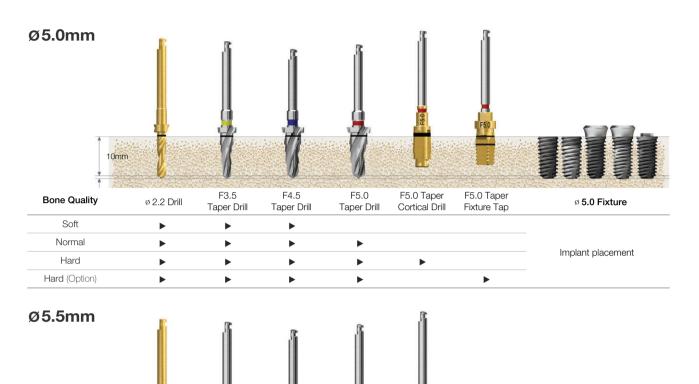

F5.5

Taper Drill

F5.5 Taper

Fixture Tap

 \blacktriangleright

Ø 5.5 Fixture

Implant placement

F3.5

Taper Drill

ø 2.2 Drill

Bone Quality

Soft

Normal

Hard

F4.5

Taper Drill

F5.0 taper cortical drill marker: Drill to the bottom of the marker for fixture lengths ≥6mm, the middle for fixture lengths ≥7mm, and the top for ≥8.5mm

Recommended insertion torque: ≤40Ncm, for the TSIII/SSIII HA: ≤35Ncm (the HA coating can fracture and flake off when placed in hard bone)


TS fixture insertion depth: In normal/hard bone should be inserted 1mm subcrestal. In soft bone, it should be inserted at bone level for higher stability.


The fixture tap should used for hard bone. Recommended speed is 25rpm or hand torgue using a mount extension and torque wrench. (F5.0 fixture tap, the bottom of the marker is fixture lengths ≤7mm and the top of the marker is for ≥8.5mm).


Drilling Sequence **Straight Drill**

TSIII CA/SA/BA/HA | SSIII CA/SA/HA | USIII CA/SA

(Length: 10mm)

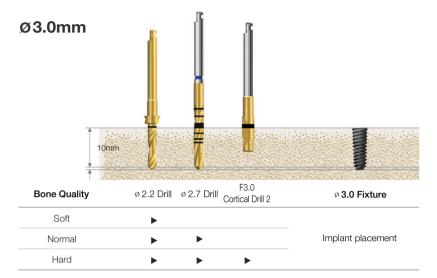


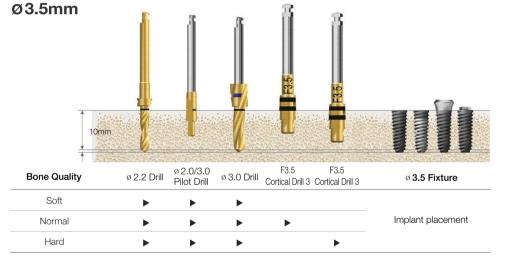


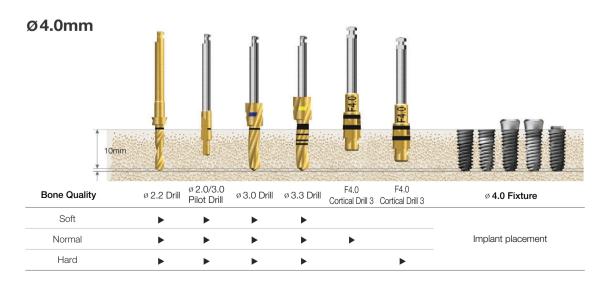


Recommended insertion torque: \leq 40Ncm, for the TSIII/SSIII HA: \leq 35Ncm (the HA coating can fracture and flake off when placed in hard bone)

TS fixture insertion depth: In normal/hard bone, it should be inserted 1mm subcrestal. In soft bone, it should be inserted at bone level for higher stability.

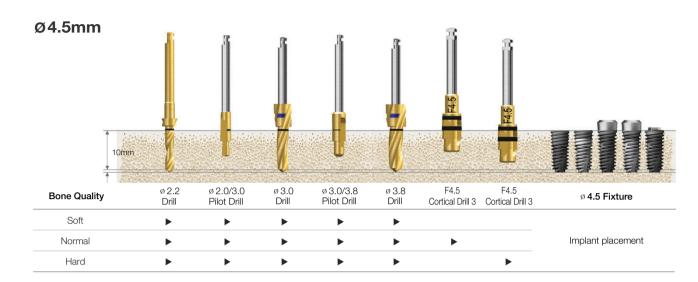

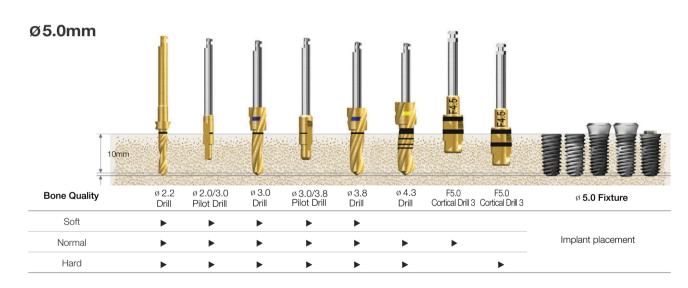


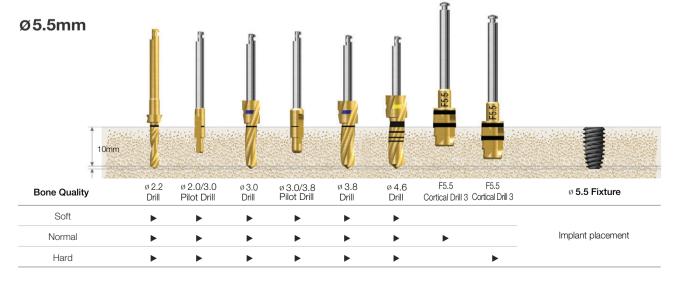



Drilling Sequence **Taper Drill**

TSIV CA/SA | USIV CA/SA

(Length : 10mm)

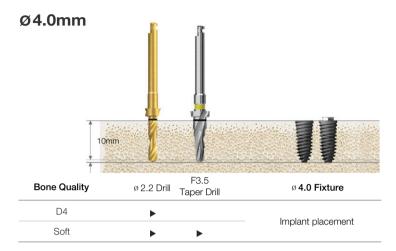

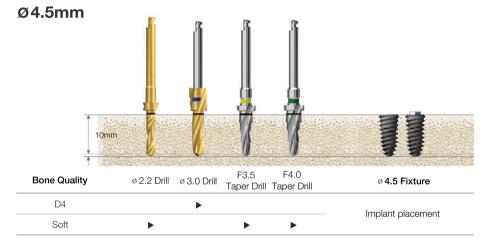

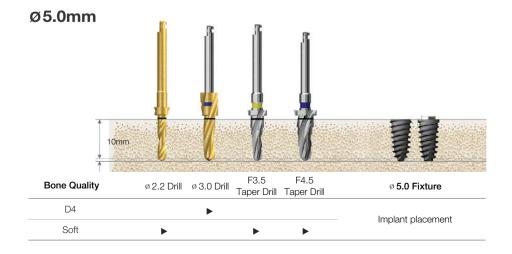




Recommended insertion torque : \leq 40Ncm


TSIV/USIV system is designed specifically for the maxillary sinus and soft bone


* Recommend reducing the insertion speed to 15rpm or lower, due to the TSIV/USIV aggresive threads.

Drilling Sequence Straight Drill

TSIV CA/SA | USIV CA/SA (Length: 10mm)

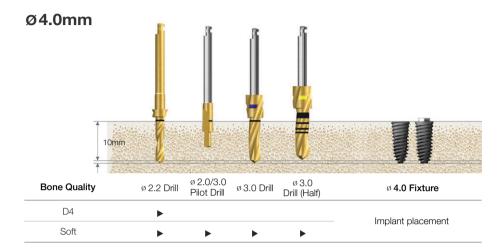

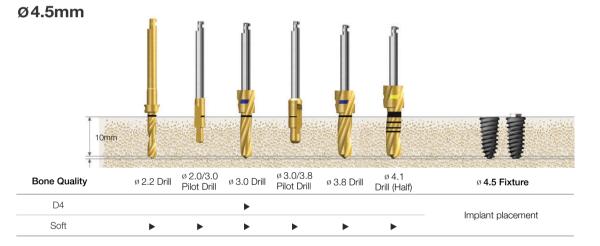

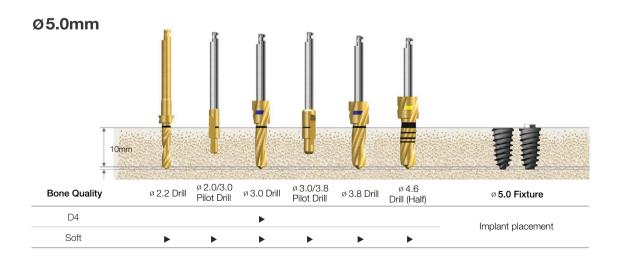



Recommended insertion torque : \leq 40Ncm


TSIV/USIV system is designed specifically for the maxillary sinus and soft bone


* Recommend reducing the insertion speed to 15rpm or lower, due to the TSIV/USIV aggresive threads.

Drilling Sequence IV Type Drill

TSIV CA/SA | USIV CA/SA

(Length: 10mm)

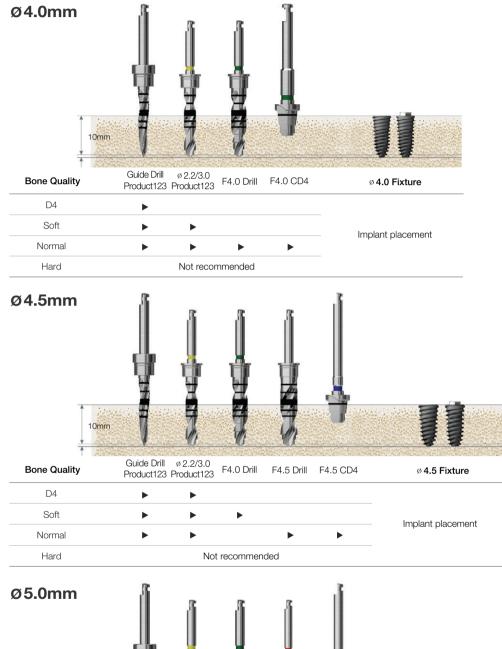

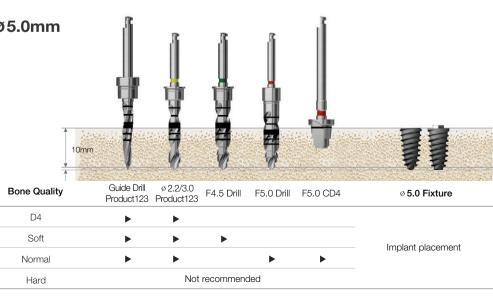



Recommended insertion torque : \leq 40Ncm

** Recommend reducing the insertion speed to 15rpm or lower, due to the TSIV/USIV aggresive threads.

Drilling Sequence Straight Drill TSII Ultre-wide CA/SA | SSII Ultre-wide CA/SA |


USII Ultre-wide CA/SA (Length: 10mm)

TSII Ultre-wide CA/SA | SSII Ultre-wide CA/SA | USII Ultre-wide CA/SA (Length: 10mm)


Ø6.0mm ø 5.2 ø 5.5 F6.0 Direct Drill Direct Drill Cortical Drill ø 3.0/3.8 Pilot Drill Bone Quality Ø 6.0 Fixture Pilot Drill Soft Normal Implant placement Hard

Ø7.0mm ø 2.2 ø 2.0/3.0 ø 3.0/3.8 ø 3.8 **Bone Quality** Drill Direct Drill Direct Drill Direct Drill Cortical Drill Soft Implant placement Normal Hard


Drilling sequence with trephine in the healed mature bone

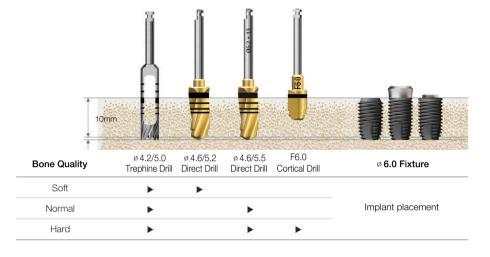


Immediate placement at the extraction socket

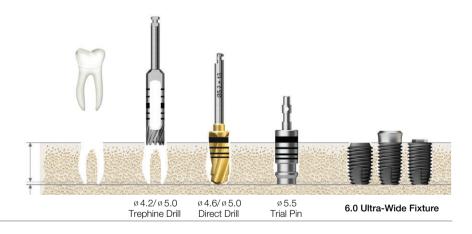

Immediate replacement of the failed implant

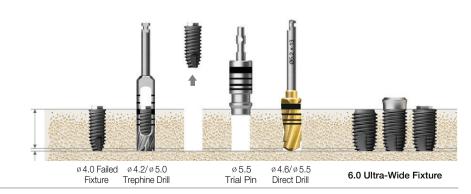

TSIII Ultre-wide CA/SA | SSIII Ultre-wide CA/SA | USIII Ultre-wide CA/SA (Length: 10mm)

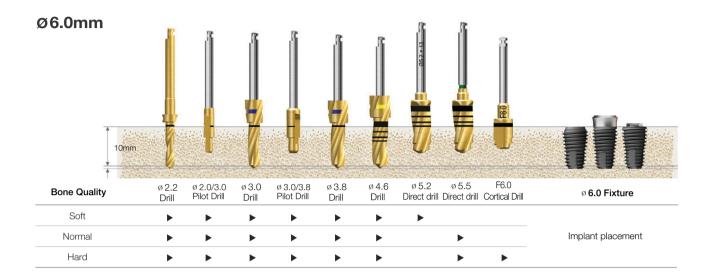





Recommended insertion torque: ≤40Ncm

**Recommend reducing the insertion speed to 15rpm or lower, due to the TSIV/USIV aggresive threads.

Drilling Sequence **Taper Drill**


TSIII Ultre-wide CA/SA | SSIII Ultre-wide CA/SA | USIII Ultre-wide CA/SA (Length: 10mm)

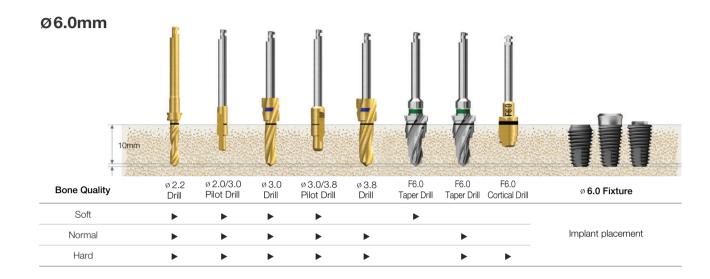


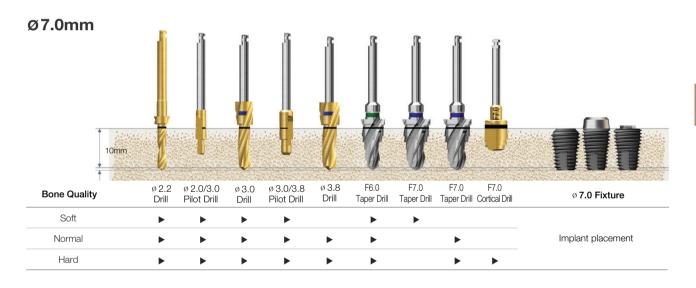

TSIV Ultre-wide CA/SA | USIV Ultre-wide CA/SA

(Length: 10mm)

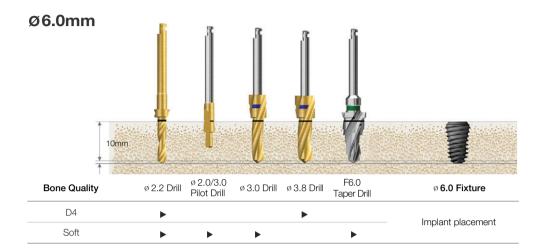


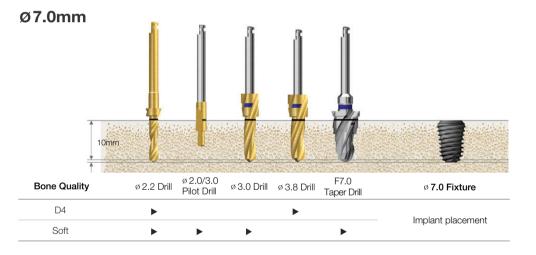


Drilling Sequence Straight Drill

TSIV Ultre-wide CA/SA | USIV Ultre-wide CA/SA

(Length: 10mm)

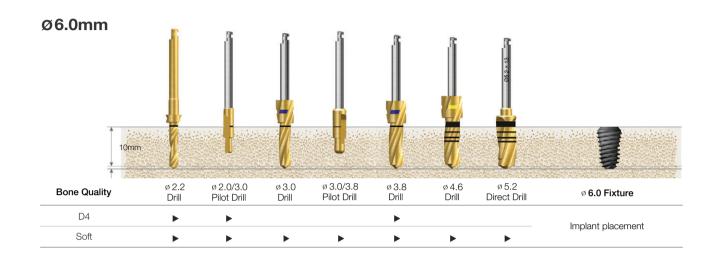


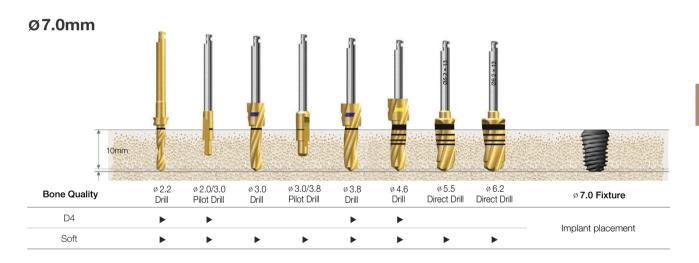



Drilling Sequence Straight Drill

MS (Length: 10mm)

Hard

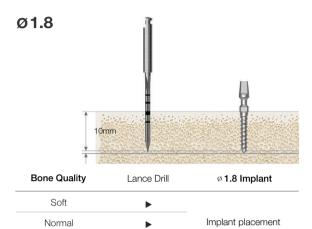

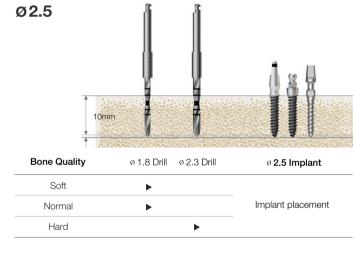

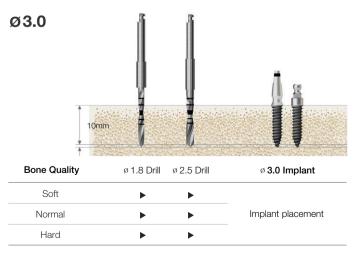

Recommended insertion torque: 3Ncm



Osstem Implant Key References

Clinic

| No. | Title | Reference / Author |
|-----|--|--|
| 1 | Retrospective clinical study of new tapered design implants in maxillary posterior areas | Oral Biology Research. 2013; 37(2):105-111 / Young-Kyun Kim et al. |
| 2 | A randomized controlled clinical trial of two types of tapered implants on immediate loading in the posterior maxilla and mandible | Int J Oral Maxillofac Implants. 2013 Nov-Dec;28(6):1602-11 (IF 1.908) / Young-Kyun Kim et al. |
| 3 | Bony window repositioning without using a barrier membrane in the lateral approach for maxillary sinus bone grafts: clinical and radiologic results at 6 months. | Int J Oral Maxillofac Implants. 2012 27:211-217 / Chang-Joo Park et al. |
| 4 | A relaxed implant bed: implants placed after two weeks of osteotomy with immediate loading: a one year clinical trial. | J Oral Implantol. 2012 Apr;38(2):155-64 / Bansal J et al. |
| 5 | A multicenter prospective study in type IV bone of a single type of implant | Implant Dent. 2012 Aug;21(4):330-34 / Su-Gwan Kim et al. |
| 6 | Comparison of clinical outcomes of sinus bone graft with simultaneous implant placement: 4-month and 6-month final prosthetic loading | Oral Surg Oral Med Oral Pathol Oral Radiol Endod. 2011 Feb;111(2):164-9 / Young-Kyun Kim et al. |
| 7 | Prospective study of tapered resorbable blasting media surface implant stability in the maxillary posterior area | Oral Surg Oral Med Oral Pathol Oral Radiol Endod. 2012 Feb 28. [Epub ahead of print] / Young-Kyun Kim et al. |
| 8 | A 1-year prospective clinical study of soft tissue conditions and marginal bone changes around dental implants after flapless implant surgery | Oral Surg Oral Med Oral Pathol Oral Radiol Endod. 2011 Jan;111(1):41-6 / Byung-Ho Choi et al. |
| 9 | Evaluation of peri-implant tissue in nonsubmerged dentallmplants: a multicenter retrospective study | Clin Implant Dent Relat Res. 2011 Dec;13(4):324-9 / Young-Kyun Kim et al. |
| 10 | A relaxed implant bed: implants placed after two weeks of osteotomy with immediate loading: a one year clinical trial | J Oral Implantol. 2012 Apr;38(2):155-64 / Bansal J et al. |
| 11 | A comparison of implant stability quotients measured using magnetic resonance frequency analysis from two directions: prospective clinical study during the initial healing period | Clin. Oral Impl. Res. 2010;21(6):591-7 / Jong-Ho Lee et al. |
| 12 | A short-term clinical study of marginal bone level change around microthreaded and platform-switched implants | J Periodontal Implant Sci. 2011;41:211-217 / Kyoo-Sung Cho et al. |
| 13 | A randomized clinical one-year trial comparing two types of nonsubmerged dental implant | Clin. Oral Impl. Res. 2010;21(2):228-36 / Jong-Ho Lee et al. |
| 14 | Short-term, multi-center prospective clinical study of short implants measuring less than 7mm | J Kor Dent Sci. 2010;3(1):11-6 / Young-Kyun Kim et al. |
| 15 | Evaluation of peri-implant tissue in nonsubmerged dentallmplants: a multicenter retrospective study | Oral Surg Oral Med Oral Pathol Oral Radiol Endod. 2009;108(2):189-95 / Young-Kyun Kim et al. |

| 16 | Evaluation of sinus bone resorption and marginal bone loss after sinus bone grafting and implant placement | Oral Surg Oral Med Oral Pathol Oral Radiol Endod. 2009;107:e21-8 / Young-Kyun Kim et al. |
|----|---|---|
| 17 | Evaluation of peri-implant tissue response according to the presence of keratinized mucosa | Oral Surg Oral Med Oral Pathol OralRadiol Endod. 2009;107:e24-8 / Young-Kyun Kim et al. |
| 18 | Study on radiographic evaluation of marginal bone loss around osseonintegrated implant after functional loading | J Kor Oral Maxillofac Surg. 2009;35:240-7 / Young - Deok, Chee |
| 19 | Four-year survival rate of RBM surface internal connection non- submerged implants and the change of the peri-implant crestal bone | J Korean Assoc Maxillofac Plast Reconstr Surg. 2009;31(3):237-42 / Sok-Min Ko et al. |

Biology

| No. | Title | Reference / Author |
|-----|---|--|
| 1 | Experiment study of bone response to hydroxyapatite coating implants: bone-implant contact and removal torque test | Oral Surg Oral Med Oral Pathol Oral Radiol. 2012 Jun 29. [Epub ahead of print] / Young-Kyun Kim et al. |
| 2 | Experimental study about the bony healing of hydroxyapatite coating implants | J Kor Oral Maxillofac Surg. 2011;27(4):295-300 / Young-Kyun Kim et al. |
| 3 | The use of autologous venous blood for maxillary sinus floor augmentation in conjunction with sinus membrane elevation: an experimental study | Clin. Oral Impl. Res. 2010;21:346-9 / Byung-Ho Choi et al. |
| 4 | Effects of soft tissue punch size on the healing of peri-Implant tissue in flapless implant surgery | Oral Surg Oral Med Oral Pathol Oral Radiol Endod. 2010;109:525-30 / Byung-Ho Choi et al. |
| 5 | Morphogenesis of the peri-implant mucosa: a comparison between flap and flapless procedures in the canine mandible | Oral Surg Oral Med Oral Pathol Oral Radiol Endod. 2009;107:66-70 / Byung-Ho Choi et al. |
| 6 | A comparative study of two noninvasive techniques to evaluate implant stability: periotest and osstell mentor | Oral Surg Oral Med Oral Pathol Oral Radiol Endod. 2009;107:513-8 / Su-Gwan Kim et al. |
| 7 | Influence of abutment connections and plaque control on the initial healing of prematurely exposed implants: an experimental study in dogs | J Periodontol. 2008;79(6):1070-4 / Byung-Ho Choi et al. |
| 8 | Er:YAG laser irradiated implant surface observation with scanning electron microscopy | J Korean Assoc Maxillofac Plast Reconstr Surg. 2008;30(6):540-5 / Seung-Ki Min et al. |
| 9 | The effect of surface treatment of the cervical area of implant on bone regeneration in mini-pig | J Kor Oral Maxillofac Surg. 2008;34:285-92 / Hong-Ju Park et al. |

| 10 | Histologic and histomorphometric evaluation of early and immediately loaded implants in the dog mandible | J Biomed Mater Res A. 2008;86:1122-7 / Su-Gwan Kim et al. |
|----|--|--|
| 11 | Effects of different depths of gap on healing of surgically created coronal defects around implants in dogs: a pilot study | J Periodontol. 2008;79(2):355-61 / June-Sung Shim et al. |
| 12 | Comparative study of removal effect on artificial plaque from RBM treated implant | J Korean Assoc Maxillofac Plast Reconstr Surg. 2007;29(4):309-20 / Hee-Jyun Oh et al. |

Biomechanics

| No. | Title | Reference / Author |
|-----|---|---|
| 1 | Evaluation of the correlation between insertion torque and primary stabilityof dental implants using a block bone test | J Periodontal Implant Sci. 2013;43:41-46 / Ki-Tae Koo et al. |
| 2 | Self-cutting blades and their influence on primary stability of tapered dental implants in a simulated low-density bone model: a laboratory study | Oral Surg Oral Med Oral Pathol Oral Radiol Endod. 2011;112:573-580 / Young-Jun Lim et al. |
| 3 | Variation in the total lengths of abutment/implant assemblies generated with a function of applied tightening torque in external and internal implant-abutment connection | Clin. Oral Impl. Res. 2011;22:834-9 / Ki-Seong Kim et al. |
| 4 | Effect of impression coping and implant angulation on the accuracy of implant impressions: an in vitro study | J Adv Prosthodont. 2010;2(4):128-33 / Seung-Geun Ahn et al. |
| 5 | Influence of implant diameter and length changes on initial stability | J Kor Acad Prosthodont. 2009;47:335-41 / Chang-Mo Jeong et al. |
| 6 | Mechanical strength of zirconia abutment in implant restoration | J KASFO. 2009;25(4):349-60 / Young-Chan Jeon et al. |
| 7 | Heat transfer to the implant-bone interface during preparation of zirconia/alumina complex abutment | Int J Oral Maxillofac Implants. 2009;24(4):679-83 / Yong-Geun Choi et al. |
| 8 | Fatigue fracture of different dental Implant system under cyclic loading | J Kor Acad Prosthodont. 2009;47(4):424-34 / In-Ho Cho et al. |
| 9 | Effect of tightening torque on abutment-fixture joint stability using 3-dimensional finite element analysis | J Kor Acad Prosthodont. 2009;47(2):125-35 / Chang-Mo Jeong et al. |
| 10 | The effect of various thread designs on the initial stability of taper implants | J Adv. Prosthodont. 2009;1:19-25 / Young-Jun Lim et al. |
| 11 | Influence of tungsten carbide/carbon coating of implant-abutment screw on screw loosening | J Kor Acad Prosthodont. 2008;46(2):137-47 / Chang-Mo Jeong et al. |

User Manual 2013.02 ver.4.0 "Disposable, re-use prohibited, medical appliance"

Osstem Implant product information

Osstem Implant's dental fixtures and products are manufactured using medical grade Titanium. Osstem Implant's abutments, denture material and surgical tools are only compatible with Osstem fixtures. For more details about any individual product, please refer to the user manual or catalog, or visit our company website (www.osstem.com). Please check all product labels for product codes. specifications, date of manufacture and expiration date.

Sterility

Fixtures, cover screws and healing abutments are cleansed and gamma-sterilized. These products are disposable sterile medical appliances, and must be used in a sterile field. If the package is damaged or has expired, it must not be used. If the product package has been opened but not used, there is a risk of contamination and it is not recommended that the product resterilized and therefore should be discarded.

Storage conditions

Store all products in a dry place at room temperature (30oC). Avoid direct sunlight.

General precautions

Dental implant surgery requires proper and formal training and education.

Cautions before dental surgery

Before dental implant surgery, a thorough patient health history review, and oral radiographic examinations must be completed to determine bone quality and proper treatment planning.

Cautions during dental implant surgery

Osstem Implant Systems are for single or two stage dental implant procedures. In order to minimize damage to the patient's tissue, special attention to temperature, surgical lesions and eliminating all sources of contamination and infection are needed. Any deviation from the standard surgical protocol increases the risk of failure. When inserting the dental implant, sufficient cooling must be introduced (water or saline) and excessive torque (greater than 55Ncm) can result in dental implant fracture or possibly bone necrosis. Placing dental implants greater than 300 has a very high risk of implant fracture. Direct pressure to the fixture should be avoided right after surgery. Immediate or delayed loading of the fixture must be determined after proper examination of the patient's bone condition and initial

"Mini" implants or implants with a diameter less than 4.0mm are not recommended for the posterior region.

Ultra-wide dental implants are recommended for the posterior region but should not be used with angled abutments. If considering an Ultra-wide dental implant, proper radiographic evaluation must be made to determine the bone mass and potential anatomical restrictions. Short dental implants (diameter greater than 5mm and shorter than 7mm) are only used for the posterior region. The clinician must

thoroughly evaluate the patient's condition and recognized the following issues: 1) bone loss due to peri-implantitis, 2) changes to the dental implant condition. 3) proper osseointegration determined by a x-ray examination. If there is movement or if there is bone loss more than 50% removing the dental implant should be a course of action. Wide diameter implants should be performed as a two stage surgery. Sufficient healing time must be given before splinting with other implants or when loading. Immediate loading is not recommended.

Take care when placing dental implants with HA coating. The coating is prone to cracking or fracturing under high torque, therefore hard bone should be avoided and be inserted under 35Ncm of force.

CA and SOSI treated dental implants are encased in a solution to prevent the chemically treated surface from reacting with air. After removing the CA or SOSI dental implant, place the implant within 15 minutes to avoid degradation of the surface.

Warning

Improper patient selection and treatment planning may result in dental implant failure or loss of bone. Osstem Implants must not be used for purpose other than prescribed and must not be altered in any shape or form. Implant movement, bone loss, and chronic infections can result in implant failure.

Indications

Osstem Implant Systems are designed to replace a patient's tooth or teeth. They can be placed in both the maxillary and submaxillary alveolar bones and after full osseointegration can be restored prosthetically. Osstem Implant Systems offer both temporary and final prosthesis and can be retained by cement, screw, overdenture or fixed bridge.

Side effects

There are possible side effects after implant surgery (lost of implant stability, damage to dentures). These issues can be caused by the lack of bone or poor bone quality, an infection, patient's poor oral hygiene, non compliance with post op procedures, movement of the implant, degradation of surrounding tissue, or improper placement of the dental implant.

Contraindications

Patients with the following contraindications are not eligible for dental implants:

- Patients with blood clotting issues or issues with wound healing.
- Diabetic natients
- Patients that smoke or drink excessively
- Patient's with compromised immune systems due disease or chemo and radiation therapy.
- Patients with an oral infection or inflammation (improper oral hygiene or teeth
- Patients with an incurable malocclusion/arthropathia and insufficient arch space.

Manufacturer: Osstem Implant Co., Ltd. 203, Geoje-daero, Yeonje-gu, Busan, Korea TEL 82-51-850-2500 FAX 82-51-861-4693



















LOT

(8)

Do not reuse









Do not resterilize

Caution, Consult accompanying documents

DEUTSCHE OSSTEM GmbH.

Mergenthalerallee 25 65760 Eschborn, Germany +49-(0)6196-777-550

EC REP







